
Post-OP Kompressions-BH Julia

Postoperative compression bra · Soutien-gorge compressif post-op ratoire · Reggiseno compressione post-operatoria · Suti  de compress o p s-operat ria · Sujetador de compresi n postoperatoria · Postoperatieve compressie BH · Ameliyat sonrası sikiřtirma sutyen · Послеоперационный сжатия бюстгальтер · حمالة الصدر الضاغطة بعد عملية احية

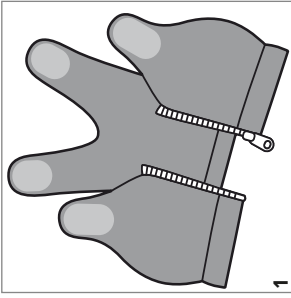1

Juzo Post-OP Indikationen

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Compression-Bra Julia

Indications

Medical product for use after surgical procedures in the chest area (with pockets for breast forms).

Description

1. The bra can be opened fully (both the fastener in the centre of the chest and at the shoulder) and is therefore easy to use and to adjust in the operating theatre and/or during recovery.
2. The shoulder straps can be adjusted with the aid of the velcro fasteners. This helps avoid skin irritation.
3. After adjusting the straps, the fastener in the centre of the chest can first be secured using the three hooks and then closed using the zipper.
4. If required, breast forms can be inserted into the pockets provided before fitting the bra.

Please note:

- Only wear your compression bra when prescribed by your physician.
- Should you have any questions regarding the use or correct fitting of the bra, please consult your physician or medical products supplier.
- To avoid creating pressure points and/or obstructing the flow within the drainage tubes, make sure that all bandages and/or drainage tubes are correctly positioned under the bra.
- The bra must be taken off before radiation therapy as it contains metal parts.
- Should the product become damaged, please consult your medical products supplier. Do not repair the bra yourself – this could compromise its quality and medical efficacy.

Washing and care instructions

Please observe the care instructions on the inside of the product. Close the zipper and velcro fasteners before washing. Wash with similar coloured items and in a mesh bag at 40 °C (black) or 60 °C (white). Using chlorine-based bleaches or chlorine-based disinfectants will reduce the life and function of the product.

Materials used

For full details, please refer to the label on the inside of the bra.

Contraindications

- Should unexpected symptoms or irritation of the wound arise, please consult your physician or medical products supplier.
- Skin disorders or skin irritations;

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Juzo jeteveypıruđos

PT

open wounds in the treated body region should be covered in a sterile manner.

Side effects

There are no known side effects when used as directed. However, if negative reactions (such as for example skin irritations) should occur while the use of our product has been prescribed, please go to your physician or medical products supplier immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use.

The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany
TYTEX A/S · Industrivej 21 · 7430 Ikast · Danmark