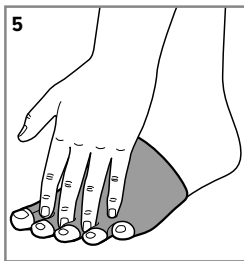
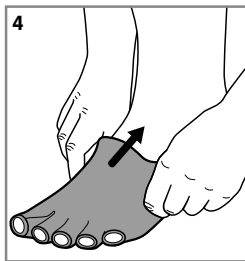
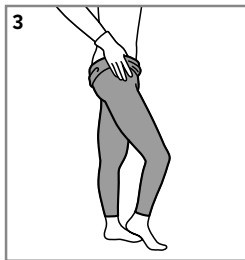
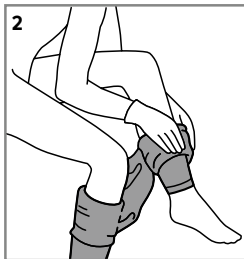
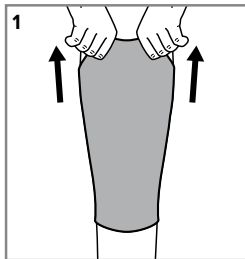
Kompressionssegmente (Bein / Fuß)

Compression segments (leg / foot)

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użyci · Инструкция по эксплуатации · تعليمات الاستخدام · Návod na používanie · Navodila za uporabo

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Intended purpose

Compression therapy involves measures to apply extensive, external pressure to treat vein problems and lymph drainage disorders. The aim of the extensive pressure is to prevent the formation of oedema and to support venous return and lymph drainage. Compression garments exert an optimal pressure gradient that gradually decreases from bottom to top. Compression therapy is primarily used to treat vein problems and oedema. It involves the use of compressive and elastic double-stretch fabrics or knitted fabrics that wrap right round the extremities. As a general rule, compression therapy is a long-term treatment requiring appropriate products that are used by the patients themselves when medically indicated and on the instructions of a doctor.

Notes for use

Your specialist retailer will be happy to show you the correct way to use the product. You can also visit [juzo.com/donning](https://www.juzo.com/donning) for video instructions relating to all Juzo donning aids. Put your compression garment on the decongested leg in the morning right after getting up. Put on and put of your compression garment while sitting down to prevent injuries or falling. Rough, unfiled finger nails or sharp-edged rings etc. may damage the very fine knitted fabric. We therefore recommend the use of Juzo special gloves. Toe nails and calluses can also cause damage, as can the defective inner lining of shoes. In order to increase the wearing comfort of your compression garments, you can hoist the silicone border a little (if there is one) a number of times during wearing.

Leg segments

1. Put one leg into the segment.
2. If applicable – put the other leg into the garment.
3. Slide the leg segment up evenly and free from creases, without twisting it. If you have pulled the garment up too high, simply push it back down and then slide the leg segment up again with less strength using the flat of your hand. Tip: fold over the silicone border before donning (if applicable).

Foot-toe-portion

4. Put your foot right into the foot-toe-portion.
5. Make sure that the knitted fabric does not crease in the areas between the toes. Adjust the individual toes if required.

Please note:

Only wear compression garments if they have been prescribed by a doctor. Compression garments should only be dispensed by properly trained staff at a medical retailer. Your Juzo compression garments are largely resistant to oils, ointments, skin moisture and other environmental influences. The durability of the knitted fabric may be affected by certain combinations of the above. This is why it is advisable to have your compression garments checked regularly at a medical retailer. For compression garments with a silicone border, please note that the legs should be depilated and free from oils, powder, creams and ointments. Please contact your specialist retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness. For hygiene reasons, this product is intended for one person only.

Washing and care instructions

Please note the care instructions on the textile label sewn into your compression garment or the printed information. Do up any fasteners (if there are any) before washing! We recommend turning compression garments inside out before washing. Wash compression garments separately when washing for the first time (they may lose colour). If possible, your compression garments should be washed every day on a gentle or delicate wash cycle (40°C). Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener! When hand washing, rinse well and do not wring. We recommend the gentle Juzo special detergent. You can reduce drying time by placing the compression garment on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang up the compression garment loosely. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Please use the delicates setting when drying in a tumble dryer (suitable for tumble-drying: see textile label or printed information on the fabric). Juzo compression garments are not suitable for dry cleaning.

Material composition

For precise details, please see the textile label sewn into your compression garment or the printed information on the fabric.

Storage information and shelf life

Store in a dry place and protect from sunlight. Standard products have a maximum shelf life of 36 months. The compression garment then has a usage period of 6 months. The usage period depends on correct handling (e.g. care, donning and doffing) and normal use of the products. The usage period of the medical product is printed on the box with an hourglass

symbol. Custom-made products are for immediate use. Body measurements may change based on the patient's condition and individual circumstances. Custom-made products are therefore designed for a usage period of 6 months. After the 6-month usage period, body measurements must be re-checked by a medical specialist. In the event of significant changes in measurements of the body part that is being treated (e.g. based on therapeutic success, weight change), re-measurement and supply of a new product may be required earlier, before the 6-month usage period comes to an end.

Indications

lymphatic oedema, lipoedema, venous congestion, post-traumatic oedema

In case of pregnancy, changing abdomen circumferences mean that a pregnancy body part should be exclusively used for treatment with pantyhose.

Following burns and for scar treatment, we recommend using our special scar therapy products.

Absolute contraindications

Advanced peripheral arterial disease (if one of these parameters applies: ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO₂ < 20 mmHg instep). Decompensated heart failure (NYHA III + IV), septic phlebitis, phlegmasia cerulea dolens, acute bacterial, viral or allergic inflammation, swelling of the extremities for unknown reasons.

Relative contraindications

Pronounced weeping dermatoses, intolerance to pressure or of any of the components of the product, severe sensory disorders of the extremities, advanced peripheral neuropathy (e.g. diabetes mellitus), primary chronic polyarthritis, peripheral arterial disease (PAD) stage III, malignant lymphoedema.

Important

If there are open wounds in the area to be treated, the affected area must be covered with an appropriate wound dressing or bandage before the compression garment can be applied. Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when using this product correctly. However, if you notice any adverse changes (e.g. irritation of the skin) during the prescribed use, please consult your doctor or medical retailer immediately. If you know that you are allergic to one or more of the product ingredients, please consult your doctor before using the product. If your

symptoms worsen during the time you are wearing the product, please take off the product and consult a medical specialist immediately. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

You can dispose of your compression garments in the usual way. There are no specific disposal instructions.

Additional information for products containing silver

As silver can deduct heat and electricity, please avoid the direct contact with electric sources, Juzo Expert Silver / Juzo Expert Strong Silver should not come in contact with metals (e.g. metal clothes hangers) when the textile is wet, the worked in silver could be discoloured thereby, the tanning agents used in the manufacture of the leather / man-made shoe lining may cause discoloration of the stocking and the silver thread in the foot area.

Additional information about the Dip Dye Collection

Each Juzo compression product with a Dip Dye design is unique. The colour gradient of the individual items is unique and cannot be reproduced exactly. Owing to the manufacturing process, slight colour variations are possible.

Additional information about the Batik Collection

Each Juzo compression product with a Batik design is unique. The colour pattern of the handmade individual items is unique and cannot be reproduced. There may be minor colour changes after the first wash. The characteristic "wrinkled look" originates from the production technique. It is only visible on the unworn product. Owing to the special colouring technique, these compression products are slightly longer before they have been worn than products without a Batik design. However, this does not have any influence on the fitting accuracy and the medical efficacy when worn.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · juzo.com