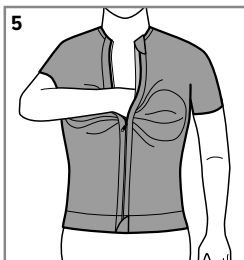
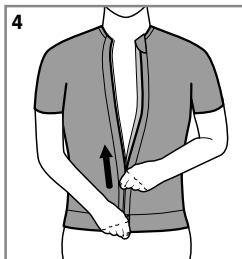
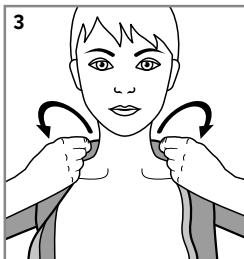
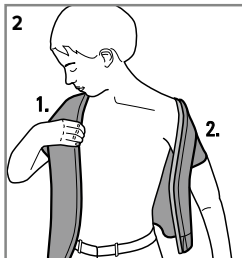
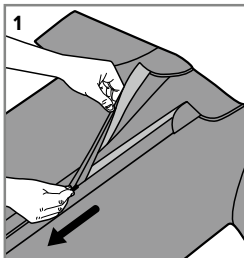
Kompressionssegmente (Thorax / Kopf)

Compression segments (thorax/head)





Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użyci · Инструкция по эксплуатации · تعليمات الاستخدام · Návod na používanie · Navodila za uporabo

A)

B)

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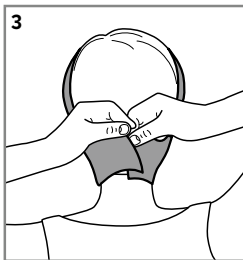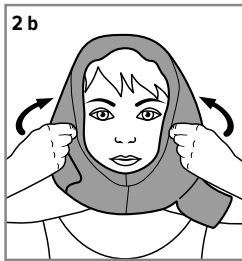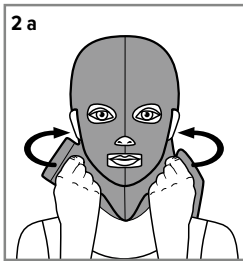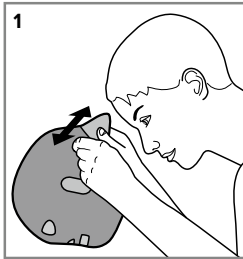Intended purpose

Compression therapy involves measures to apply extensive, external pressure to treat lymphatic drainage disorders. The aim of extensive pressure is to prevent the formation of oedema and to support lymphatic drainage. Compression therapy is primarily used to treat oedema. As a general rule, compression therapy is a long-term treatment requiring appropriate products that are used by the patients themselves when medically indicated and on the instructions of a doctor.

Notes for use

Your specialist retailer will gladly show you the correct way to apply the product. Rough, unfiled finger nails or sharp-edged rings etc. may damage the very fine knitted fabric. We therefore recommend the use of

Juzo special gloves.

A) Thorax

1. Fully open the product closure.
2. Insert one of your arms into one sleeve, then the other arm into the second sleeve.
3. Now position the product over your shoulders and adjust.
4. Do up the closure from the bottom to the top. If there are any hooks, close them first.
5. If there are cups, you should initially close the product to just above the chest. In order to ideally position each breast in the relevant cup, insert the opposite hand and lift the breast slightly. Then do up the closure completely.
6. If there is a body fastening, close it at the end. (no illustration)

B) Neck and chin support/face mask

1. Open the product and place it on your face.
2. a Face mask
 - b Neck and chin support
Pull the product over your head toward the back. Check the correct position of possible cut-outs.
3. Now close the mask at the back of your head.

Please note

Only wear compression garments if they have been prescribed by a doctor. Compression garments should only be dispensed by properly trained staff at a medical retailer. Your Juzo compression garments are largely resistant to oils, ointments, skin moisture and other environmental influences. The durability of the knitted fabric may be affected by certain combinations of the above. This is why it is advisable to have your compression garments checked regularly at a medical retailer. Please

contact your specialist retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness. For hygiene reasons, this product is intended for one person only.

Washing and care instructions

Please note the care instructions on the textile label sewn into your compression garment or the printed information. Do up any fasteners (if there are any) before washing! We recommend turning compression garments inside out before washing. Wash compression garments separately when washing for the first time (they may lose colour). If possible, your compression garments should be washed every day on a gentle or delicate wash cycle (40°C). Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener! When hand washing, rinse well and do not wring. We recommend the gentle Juzo special detergent. You can reduce drying time by placing the compression garment on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang up the compression garment loosely. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Please use the delicates setting when drying in a tumble dryer (suitable for tumble-drying: see textile label). Juzo compression garments are not suitable for dry cleaning.

Material composition

For precise details, please see the textile label sewn into your compression garment or the printed information on the fabric.

Storage information and shelf life

Store in a dry place and protect from sunlight. Custom-made products are for immediate use. Body measurements may change based on the patient's condition and individual circumstances. Custom-made products are therefore designed for a usage period of 6 months. After the 6-month usage period, body measurements must be re-checked by a medical specialist. In the event of significant changes in measurements of the body part that is being treated (e.g. based on therapeutic success, weight change), re-measurement and supply of a new product may be required earlier, before the 6-month usage period comes to an end.

Indications

Lymphatic oedema, lipoedema, post-traumatic oedema

Following burns and for scar treatment, we recommend using our special scar therapy products.

Absolute contraindications

Advanced peripheral arterial disease (if one of these parameters applies: ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO₂ < 20 mmHg instep). Decompensated heart failure (NYHA III + IV), septic phlebitis, phlegmasia cerulea dolens, acute bacterial, viral or allergic inflammation, swelling of the extremities for unknown reasons.

Relative contraindications

Pronounced weeping dermatoses, intolerance to pressure or of any of the components of the product, severe sensory disorders of the extremities, advanced peripheral neuropathy (e.g. diabetes mellitus), primary chronic polyarthritis, peripheral arterial disease (PAD) stage I/II, malignant lymphoedema.

Important

If there are open wounds in the area to be treated, the affected area must be covered with an appropriate wound dressing or bandage before the compression garment can be applied. Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your symptoms worsen during the time you are wearing the product, please take off the product and consult a medical specialist immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

You can dispose of your compression garments in the usual way. There are no specific disposal instructions.

Additional information for products containing silver

As silver can deduct heat and electricity, please avoid the direct contact with electric sources. Juzo Expert Silver should not come in contact with metals (e.g. metal clothes hangers) when the textile is wet, the worked in silver could be discoloured thereby.

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