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## Purpose

The consistent application of uniform surface pressure by the compression garments has a beneficial effect on scar maturation and development. Therapeutic application of compression garments results in decreased blood circulation and increased tissue elasticity. The aim is to reduce the risk of the formation of hypertrophic scars and/or keloids or achieve a reduction in their appearance. The objective of compression therapy is to flatten the scar, increase tissue elasticity and reduce scar rigidity.

## Notes for application

Your specialist retailer will gladly show you the correct way to apply the product. Wear your compression garments for 23 hours non-stop (or as advised by your doctor). Compression garments offer a certified UPF 80 UV protection factor in accordance with UV Standard 801. The UV protection only applies to those body regions that are covered with compression garments. Rough, unfiled finger nails or sharp-edged rings etc. may damage the very fine knitted fabric. Therefore please use Juzo Special Gloves. Toe nails and calluses can cause damage, as can defective inner lining of shoes. For compression garments with a silicone border, please note: legs should be depilated and free from oils, powder, creams and ointments. In order to increase the wearing comfort of your compression garments, you can hoist the silicone border a little (if there is one) a number of times during wearing. Put on your products for the lower extremities while sitting down to prevent injuries or falling.

## Leg segments

1. Put one leg into the segment.
2. If applicable - put the other leg into the garment.
3. Slide the leg segment up evenly and free from creases, without twisting it. If you have pulled the garment up too high, simply push it back down and then slide the leg segment up again with less strength using the flat of your hand. Tip: fold over the silicone border before donning (if applicable).

## Foot-toe-portion

4. Put your foot right into the foot-toe-portion.
5. Make sure that the knitted fabric does not crease in the areas between the toes. Adjust the individual toes if required.

## Please note:

Only wear compression garments if they have been prescribed by a doctor. Compression garments should only be dispensed by properly trained staff at a medical retailer. Progressing the success of treatment or a change in condition means that the compression garments will have to be remeasured, and, if necessary, new items will need to be made. With products for small children, the fit of the compression garments should be checked more regularly. Your compression garments are largely resistant to oils,

ointments, skin moisture and other environmental influences. The durability of the knitted fabric may be affected by certain combinations of the above. This is why it is advisable to have your compression garments checked regularly at a medical retailer. Please contact your specialist retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness. For hygiene reasons, this product is intended for one person only.

## Washing and care instructions

Please note the care instructions on the textile label sewn into your compression garment. In order to guarantee hygiene and material regeneration, where possible compression garments should be washed daily if they are worn regularly. Do up any fasteners (if there are any) before washing! We recommend turning compression stockings with silicone borders inside out before washing. Wash compression garments separately when washing for the first time (they may lose colour). Your compression garments should be washed every day on a gentle or delicate wash cycle (40 °C). Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener. When hand washing, rinse well and do not wring. We recommend the gentle Juzo Special Detergent. You can reduce drying time by placing the compression garment on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang up the compression garment loosely. Do not leave it in the towel and do not dry on a radiator or in the sun. Compression garments are not suitable for dry cleaning or tumble drying.

## Material composition

For precise details, please see the textile label sewn into your compression garment.

## Storage information and shelf life

Store in a dry place and protect from sunlight. Standard products have a maximum shelf life of 36 months. The compression garment then has a usage period of 6 months. The usage period depends on correct handling (e.g. care, donning and doffing) and normal use of the products. The usage period of the medical product is printed on the package with an hourglass symbol. Custom-made products are for immediate use. Body measurements may change based on the patient's condition and individual circumstances. Custom-made products are therefore designed for a usage period of 6 months. After the 6-month usage period, body measurements must be re-checked by a medical specialist. In the event of significant changes in measurements of the body part that is being treated (e.g. based on therapeutic success, weight change), re-measurement and supply of a new product may be required earlier, before the 6-month usage period comes to an end.

## Indications

Damage to the skin and healing for skin with hyper-

trophic scars and keloids.

After injuries/burns/operations, to improve scar maturation until the scar is completely healed, to stabilise the surgical outcome. For prevention if there is a tendency towards keloids and hypertrophic scar formation.

### **Absolute contraindications**

Advanced peripheral arterial disease (if one of these parameters applies: ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO<sub>2</sub> < 20 mmHg instep). Decompensated heart failure (NYHA III + IV), septic phlebitis, phlegmasia cerulea dolens, acute bacterial, viral or allergic inflammation, swelling of the extremities for unknown reasons.

### **Relative contraindications**

Pronounced weeping dermatoses, intolerance to pressure or of any of the components of the product, severe sensory disorders of the extremities, advanced peripheral neuropathy (e.g. diabetes mellitus), primary chronic polyarthritis, peripheral arterial disease (PAD) stage I/II, malignant lymphoedema.

### **Important**

If there are open wounds in the area to be treated, the affected area must be covered with an appropriate wound dressing or bandage before the compression garment can be applied.

For chronic lymphoedema or lipoedema, we recommend our lymphological products.

Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

### **Side effects**

There are no known side effects when using this product correctly. However, if you notice any adverse changes (e.g. irritation of the skin) during the prescribed use, please consult your doctor or medical retailer immediately. If you know that you are allergic to one or more of the product ingredients, please consult your doctor before using the product. If your symptoms worsen during the time you are wearing the product, please take off the product and consult a medical specialist immediately. The manufacturer does not accept any liability for damage / injuries resulting from improper use or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

### **Disposal**

You can dispose of your compression garments in the usual way. There are no specific disposal instructions.

### **Additional information for products containing silver**

As silver can deduct heat and electricity, please avoid the direct contact with electric sources. Juzo ScarComfort Silver should not come in contact with metals (e. g. metal clothes hangers) when the textile is wet. The worked in silver could be discoloured thereby. The tanning agents used in the manufacture of the leather / man-made shoe lining may cause discoloration of the compression product and the silver thread in the foot area.

### **Additional information for Batik Collection**

Each Juzo compression product with a Batik design is unique. The colour pattern of the handmade individual items is unique and cannot be reproduced. There may be minor colour changes after the first wash. The characteristic "wrinkled look" originates from the production technique. It is only visible on the unworn product. Owing to the special colouring technique, these compression products are slightly longer before they have been worn than products without a Batik design. However, this does not have any influence on the fitting accuracy and the medical efficacy when worn.

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