Juzo Silon[®]-TEX

Hinweise · Notices

Conseils · Avvertenze · Instruções · Pautas · Instructies · Bruksanvisning · Informationer · Talimati · Указание · ارشادات

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Notes for using compression garments with integrated Juzo Silon®-TEX

Scars are caused by injuries or surgical procedures. An important component of scar treatment is the use of silicone in combination with compression garments.

Combination of compression and silicone

The ideal treatment of scars is a combination of silicone and compression. Combining the compression garment's consistent surface pressure and silicone has a positive healing effect on the scar. In addition, functionality and appearance of the scarred area are improved.

Silicone

Medical silicone allows a moist environment to develop on the scar, meaning the scar doesn't dry out. It keeps the scar soft and has a beneficial effect on the scar tissue.

Product description

Silon®-TEX is composed of a thin medical silicone layer applied on a soft and supple textile base material. Silon®-TEX allows individual integration into the compression garment depending on the medical requirements. Here, the following must be observed:

- as a basic rule, Silon®-TEX is not incorporated in the entire circumference of the product
- It can be incorporated up to a maximum of 50% of the circumference (of the affected body part) because otherwise, the compressive effect of the knitted fabric would increase to an uncontrollable extent.
- Silon®-TEX can only be used in combination with Juzo ScarComfort compression knits.
- Please consider fastening systems for easier donning.

Notes for use

Your specialist medical retailer will be happy to show you the correct way to use the product.

- The scar and the surrounding skin must be thoroughly cleaned.
- The scar tissue should be kept dry and free of oils.
- Do not use ointments, creams, gels or lotions in combination with Silon®-TEX.
- When putting on the compression garment, please make sure the Silon®-TEX is placed directly on the scarred skin without creases.
- Do not use on irritated skin.
- Do not use in case of known intolerance to the component (ultra-pure silicone).
- If skin intolerances occur during use, please stop using the product and consult your doctor.
- Compression garments with silicone should be used according to your doctor's instructions, for a maximum of 23 hours per day.
- Do not use Silon®-TEX on open wounds!

Duration of use

In terms of medical effectiveness (compression effect), compression garments with integrated Silon®-TEX are generally intended to be worn for 6 months. This period depends on correct handling (e.g. care, donning and doffing) and normal use of the products.

Washing and care instructions

Please note the care instructions on the textile label sewn into your compression garment. In order to guarantee hygiene and material regeneration, compression garments should be washed daily.

Material composition

Silicone layer: 100% medical silicone

Base material: 81% polyamide, 19% elastane

Indications

- damage to the skin and healing for skin with hypertrophic scars and keloids
- after injuries / burns / operations
- for prevention if there is a tendency towards keloids and hypertrophic scar formation

Contraindications

- Do not use if you have a transplant that is not firmly established.
- **Do not use Silon®-TEX on cracked skin or open wounds**

Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when using this product correctly. However, if you notice any adverse changes such as a skin allergy based on sensitivity to silicone, or itching, please consult your doctor or medical retailer immediately. If you know that you are intolerant to one or more of the product's components, please consult your doctor before using the product. If your symptoms worsen during the time you are wearing the product, please consult your doctor immediately. If the skin irritation or skin allergy continues, please stop using Silon®-TEX. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

Disposal

You can dispose of your compression garments in the usual way. There are no specific disposal instructions.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany