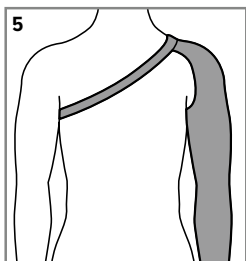
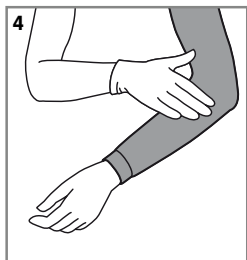
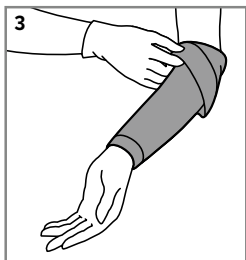
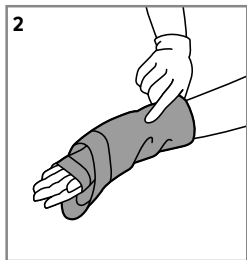
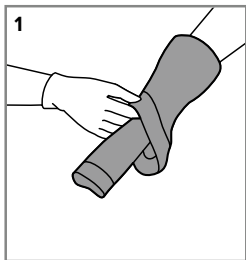
Kompressionsärmel

Compression sleeves

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Kullanım talimatları · Upute za uporabu · Инструкция по эксплуатации · تعليمما الاستخدام · Navodila za uporabo

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Intended purpose

Compression therapy involves measures to apply extensive, external pressure to treat vein problems and lymph drainage disorders. The aim of the extensive pressure is to prevent the formation of oedema and to support venous return and lymph drainage. Compression garments exert an optimal pressure gradient that gradually decreases from bottom to top. Compression therapy is primarily used to treat vein problems and oedema. It involves the use of compressive and elastic double-stretch fabrics or knitted fabrics that wrap right round the extremities. As a general rule, compression therapy is a long-term treatment requiring appropriate products that are used by the patients themselves when medically indicated and on the instructions of a doctor.

Notes for use

Your specialist retailer will gladly show you the correct way to apply the product. You can also visit www.juzo.com/donning for video instructions relating to all Juzo donning aids. Put on your compression garments in the morning straight after you get out of bed. Rough, unfiled finger nails or sharp-edged rings etc. may damage the very fine knitted fabric. We therefore recommend the use of **Juzo special gloves**.

Compression sleeves with knitted edges or silicone borders

1. Pull the upper half of the sleeve over the lower half of the sleeve.
2. Roll the sleeve over your hand and continue up your arm until the rolled-over section reaches your elbow.
3. Work the rolled section of the sleeve over your elbow and roll it towards your shoulder bit by bit.
4. Spread out the sleeve's knitted fabric so that there are no creases and it is securely fitted over the entire length of the arm. If the sleeve has been put on properly, it should have no creases and should not restrict movement.

Compression sleeves with shoulder cap and strap or bra fastening

Put one part of the shoulder strap across your chest and the other over your back. Next, fasten both ends together. Please bear in mind that you can change the length of the shoulder strap using the adjustor slide if it is too loose or too tight.

Please note

Only wear compression garments if they have been prescribed by a doctor. Compression garments should only be dispensed by properly trained staff at a medical retailer. Your Juzo compression garments are largely resistant to oils, ointments, skin moisture and other environmental influences. The durability of the knitted fabric may be affected by certain combinations of the above. This is why it is advisable to have your compression garments checked regularly at a medical retailer. For compression garments with a silicone border, please note that the arms should be depilated and free from oils, powder, creams and ointments. Please contact your specialist retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness. For hygiene reasons, this product is intended for one person only.

Washing and care instructions

Please note the care instructions on the textile label sewn into your compression garment or the printed information. Do up any fasteners (if there are any) before washing! We recommend turning compression garments inside out before washing. Wash compression garments separately when washing for the first time (they may lose colour). If possible, your compression garments should be washed every day on a gentle or delicate wash cycle (40°C). Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener! When hand washing, rinse well and do not wring. We recommend the gentle Juzo special detergent. You can reduce drying time by placing the compression garment on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang up the compression garment loosely. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Please use the delicates setting when drying in a tumble dryer (suitable for tumble-drying: see textile label). Juzo compression garments are not suitable for dry cleaning.

Material composition

For precise details, please see the textile label sewn into your compression garment.

Storage information and shelf life

Store in a dry place and protect from sunlight. Standard products have a maximum shelf life of 36 months. The compression garment then has a usage period of 6 months. The usage period depends on correct handling (e.g. care, donning and doffing) and normal use of the products. The usage period of the medical product is printed on the box with an hourglass symbol. Custom-made products are for immediate use. Body measurements may change based on the patient's condition and individual circumstances. Custom-made products are therefore designed for a usage period of 6 months. After the 6-month usage period, body measurements must be re-checked by a medical specialist. In the event of significant changes in measurements of the body part that is being treated (e.g. based on therapeutic success, weight change), re-measurement and supply of a new product may be required earlier, before the 6-month usage period comes to an end.

Indications

- Lymphoedema
- Venous congestion
- Post-traumatic swelling

Following burns and for scar treatment, we recommend using our special scar therapy products.

Absolute contraindications

Decompensated heart failure, peripheral artery disease (PAD) stage III/IV, erosive dermatoses, acute severe erysipelas, septic phlebitis, phlegmasia cerulea dolens, acute bacterial, viral or allergic inflammation, untreated or advanced skin diseases/contact eczema, untreated open wounds, swelling of the extremity of unknown cause, untreated malignant tumours.

Relative contraindications

Peripheral artery disease (PAD) stage I/II, skin diseases (e.g. blistering dermatoses), malignant lymphoedema, skin infections, severe sensory disorders of the extremities, advanced peripheral neuropathy (e.g. diabetes mellitus), compensated heart failure, rheumatoid arthritis, intolerance to pressure or ingredients of the product. If there are open wounds in the area to be treated, the affected area has to be covered with an appropriate wound dressing or bandaging before the sleeve can be applied. Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your compression garment can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany