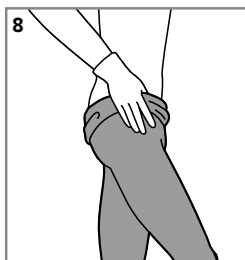
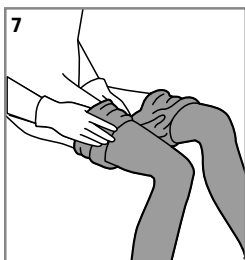
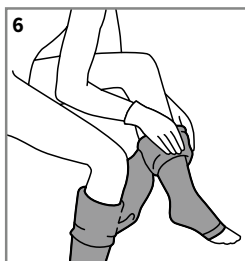
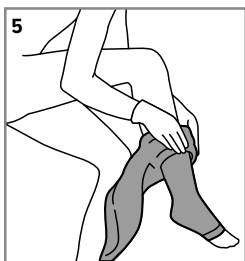
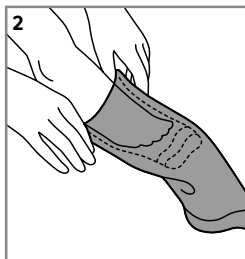
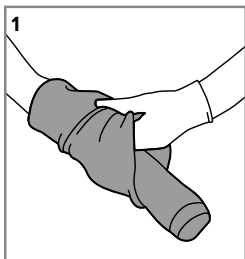
Bas et collants de compression

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Intended purpose

Compression therapy involves measures to apply extensive, external pressure to treat vein problems and lymph drainage disorders. The aim of the extensive pressure is to prevent the formation of oedema and to support venous return and lymph drainage. Compression garments exert an optimal pressure gradient that gradually decreases from bottom to top. Compression therapy is primarily used to treat vein problems and oedema. It involves the use of compressive and elastic double-stretch fabrics or knitted fabrics that wrap right round the extremities. As a general rule, compression therapy is a long-term treatment requiring appropriate products that are used by the patients themselves when medically indicated and on the instructions of a doctor.

Notes for use

For compression stockings/tights with open toes, use the Juzo Slippie donning aid. Your specialist retailer will be happy to show you the correct way to use the product. You can also visit www.juzo.com/donning for video instructions relating to all Juzo donning aids. Put on your compression garment in the morning straight after getting up. Put on and put off your compression garment while sitting down to prevent injuries or falling. Rough, unfiled finger nails or sharp-edged rings etc. may damage the very fine knitted fabric. We therefore recommend the use of Juzo special gloves. Toe nails and calluses can also cause damage, as can the defective inner lining of shoes.

Compression stockings

1. Pull the stocking over your forearm and hold on to the foot section from the inside with your fingers. Turn the stocking inside out by pulling it out as far as the heel.
2. Using both hands, stretch open the opening and pull the foot section over your foot as far as the instep. Make sure that the heel fabric is sitting on the heel.
3. Gradually pull the stocking further over your foot – as you do so, you will also turn it back the right way out.
4. Now pull up the stocking evenly and free from any wrinkles – without twisting it or pulling at the edges – and at the same time apply it to your leg with the flat of your hand. If you have pulled the stocking up too high, simply push it back down to the ankle and then pull it up again with less strength. Tip: fold over the silicone border before donning (if applicable).

Compression tights

5. Turn inside out as far as the heel, slip in the first leg as described in steps 2 and 3 and fit to below the knee.
6. Repeat the procedure with your other leg.
7. Once you have put the tights on both legs to below the knees, turn both sides the right way out again and pull them up both legs bit by bit. Then adjust the sit of the knitted fabric with the flat of your hand.
8. Now pull the body part up to your waist. Make sure that the gusset is positioned vertically in the crotch. Please bear in mind that your compression tights must never be overstretched. Distribute the knitted fabric in stages from the bottom to the top.

Please note:

Only wear compression garments if they have been prescribed by a doctor. Compression garments should only be dispensed by properly trained staff at a medical retailer. Your Juzo compression garments are largely resistant to oils, ointments, skin moisture and other environmental influences. The durability of the knitted fabric may be affected by certain combinations of the above. This is why it is advisable to have your compression garments checked regularly at a medical retailer. For stockings with a silicone border, please note that legs should be depilated and free from oils, powder, creams and ointments. Please contact your specialist retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness. For hygiene reasons, this product is intended for one person only.

Washing and care instructions

Please note the care instructions on the textile label sewn into your compression garment or the printed information. Do up any fasteners (if there are any) before washing! We recommend turning compression garments inside out before washing. Wash compression garments separately when washing for the first time (they may lose colour). If possible, your compression garments should be washed every day on a gentle or delicate wash cycle (40°C). Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener! When hand washing, rinse well and do not wring. We recommend the gentle Juzo special detergent. You can reduce drying time by placing the compression garment on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang up the compression garment loosely. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Please use the delicates setting when drying in a tumble dryer (suitable for tumble-drying: see textile label or printed information on the fabric). Juzo compression garments are not suitable for dry cleaning.

Material composition

For precise details, please see the textile label sewn into your compression garment or the printed information on the fabric.

Storage information and shelf life

Store in a dry place and protect from sunlight. Standard products have a maximum shelf life of 36 months. The compression garment then has a usage period of 6 months. The usage period depends on correct handling (e.g. care, donning and doffing) and normal use of the products. The usage period of the medical product is printed on the box with an hourglass symbol. Custom-made products are for immediate use. Body measurements may change based on the patient's condition and individual circumstances. Custom-made products are therefore designed for a usage period of 6 months. After the 6-month usage period, body measurements must be re-checked by a medical specialist. In the event of significant changes in measurements of the body part that is being treated (e.g. based on therapeutic success, weight change), re-measurement and supply of a new product may be required earlier, before the 6-month usage period comes to an end.

Indications

Primary and secondary varicose veins, varicose veins during pregnancy, to support sclerotherapy, following vein surgery procedures, thrombophlebitis (superficial) and status post healed phlebitis, deep vein thrombosis of the leg, status post thrombosis, post-thrombotic syndrome, prevention of thrombosis in mobile patients, CVI of stages I – III according to Widmer or C1 – C6 according to CEAP, ulcer prevention, deep venous insufficiency, angiodysplasia, lymphoedema, oedema during pregnancy, post-traumatic oedema, postoperative oedema, idiopathic cyclical oedema, lipoedema, lipo-lymphoedema, phlebolympoedema, congestion conditions following immobility (arthrogenic congestion syndrome, paresis and partial paresis of the extremity), improvement of venous symptoms, occupational oedema (sedentary and standing jobs).

In case of pregnancy, changing abdomen circumferences mean that a pregnancy body part should be exclusively used for treatment with pantyhose.

Following burns and for scar treatment, we recommend using our special scar therapy products.

Absolute contraindications

Advanced peripheral arterial disease (if one of these parameters applies: ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO₂ < 20 mmHg instep). Decompensated heart failure (NYHA III + IV), septic phlebitis, phlegmasia cerulea dolens, acute bacterial, viral or allergic inflammation, swelling of the extremities for unknown reasons.

Relative contraindications

Pronounced weeping dermatoses, intolerance to pressure or of any of the components of the product, severe sensory disorders of the extremities, advanced peripheral neuropathy (e.g. diabetes mellitus), primary chronic polyarthritis, peripheral arterial disease (PAD) stage I/II, malignant lymphoedema.

Important

If there are open wounds in the area to be treated, the affected area must be covered with an appropriate wound dressing or bandage before the compression garment can be applied. Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when using this product correctly. However, if you notice any adverse changes (e.g. irritation of the skin) during the prescribed use, please consult your doctor or medical retailer immediately. If you know that you are allergic to one or more of the product ingredients, please consult your doctor before using the product. If your symptoms worsen during the time you are wearing the product, please take off the product and consult a medical specialist immediately. The manufacturer does not accept any liability for damage / injuries resulting from improper use or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical con-

dition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

You can dispose of your compression garments in the usual way. There are no specific disposal instructions.

Additional information for products containing silver

As silver can deduct heat and electricity, please avoid the direct contact with electric sources. Juzo Expert Silver / Juzo Expert Strong Silver / Juzo Dynamic Silver should not come in contact with metals (e.g. metal clothes hangers) when the textile is wet, the worked in silver could be discoloured thereby, the tanning agents used in the manufacture of the leather / man-made shoe lining may cause discoloration of the stocking and the silver thread in the foot area.

Additional information about the Dip Dye Collection

Juzo Compression Stockings and Compression Sleeves in Dip Dye design are unique products. The colour gradient of the individual items is unique and cannot be reproduced exactly. Owing to the manufacturing process, slight colour variations are possible.

Additional information about the Batik Collection

Juzo compression stockings, compression pantyhose and compression sleeves in tie-dye design are unique products. The colour pattern of the handmade individual items is unique and cannot be reproduced. There may be minor colour changes after the first wash. The characteristic "wrinkled look" originates from the production technique. It is only visible on the unworn product. Due to the special dyeing technique the compression stockings, compression pantyhose and compression sleeves are slightly longer in unworn condition than products without tie-dye design. However, this does not have any influence on the fitting accuracy and the medical efficacy when worn.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · juzo.com