
Kompressionssegmente

Compression segments

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Kullanım talimatları · Upute za uporabu · Инструкция по эксплуатации · ارشادات · Navodila za uporabo

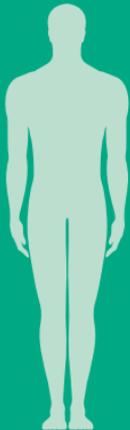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Intended purpose

Compression therapy involves measures to apply extensive, external pressure to treat vein problems and lymph drainage disorders. The aim of the extensive pressure is to prevent the formation of oedema and to support venous return and lymph drainage. Compression garments exert an optimal pressure gradient that gradually decreases from bottom to top. Compression therapy is primarily used to treat vein problems and oedema. It involves the use of compressive and elastic double-stretch fabrics or knitted fabrics that wrap right round the extremities. As a general rule, compression therapy is a long-term treatment requiring appropriate products that are used by the patients themselves when medically indicated and on the instructions of a doctor.

Notes for use

Your specialist retailer will gladly show you the correct way to apply the product. Put on your compression garments in the morning straight after you get out of bed. Rough, unfilled finger nails or sharp-edged rings etc. may damage the very fine knitted fabric. We therefore recommend the use of **Juzo special gloves**. Toe nails and calluses can cause damage, as can defective inner lining of shoes.

Please note

Only wear compression garments if they have been prescribed by a doctor. Compression garments should only be dispensed by properly trained staff at a medical retailer. Your Juzo compression garments are largely resistant to oils, ointments, skin moisture and other environmental influences. The durability of the knitted fabric may be affected by certain combinations of the above. This is why it is advisable to have your compression garments checked regularly at a medical retailer. For stockings with a silicone border, please note that legs should be depilated and free from oils, powder, creams and ointments. Please contact your specialist retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness. For hygiene reasons, this product is intended for one person only.

Washing and care instructions

Please note the care instructions on the textile label sewn into your compression garment or the printed information. Do up any fasteners (if there are any) before washing! We recommend turning compression garments inside out before washing. Wash compression garments separately when washing for the first time (they may lose colour). If possible, your compression garments should be washed every day on a gentle or delicate wash cycle (40°C). Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener! When hand washing, rinse well and do not wring. We recommend the gentle Juzo special detergent. You can reduce drying time by placing the compression garment on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang up the compression garment loosely. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Please use the delicates setting when drying in a tumble dryer (suitable for tumble-drying: see textile label). Juzo compression garments are not suitable for dry cleaning.

Material composition

For precise details, please see the textile label sewn into your compression garment.

Storage information and shelf life

Store in a dry place and protect from sunlight. Standard products have a maximum shelf life of 36 months. The compression garment then has a usage period of 6 months. The usage period depends on correct handling (e.g. care, donning and doffing) and normal use of the products. The usage period of the medical product is printed on the box with an hourglass symbol. Custom-made products are for immediate use. Body measurements may change based on the patient's condition and individual circumstances. Custom-made products are therefore designed for a usage period of 6 months. After the 6-month usage period, body measurements must be re-checked by a medical specialist. In the event of significant changes in measurements of the body part that is being treated (e.g. based on therapeutic success, weight change), re-measurement and supply of a new product may be required earlier, before the 6-month usage period comes to an end.

Indications

- lymphatic oedema
- lipoedema
- venous congestion
- post-traumatic oedema

Following burns and for scar treatment, we recommend using our special scar therapy products.

Absolute contraindications

Peripheral artery disease (PAD) stage III/IV, decompensated heart failure, septic phlebitis, phlegmasia cerulea dolens, acute erysipelas, acute bacterial, viral or allergic inflammation, untreated or advanced skin diseases/contact dermatitis, untreated open wounds, swelling of the extremity of unknown cause, untreated malignant tumours.

Relative contraindications

Pronounced weeping dermatoses, malignant lymphoedema, peripheral artery disease (PAD) stage I/II, peripheral neuropathy in association with diabetes mellitus, sensory disorders of the extremities, compensated heart failure, skin infections, skin diseases (e.g. blistering dermatoses), intolerance to pressure or the ingredients of the product, rheumatoid arthritis. If there are open wounds in the area to be treated, the affected area must be covered with an appropriate wound dressing or bandage before the compression garment can be applied. Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

You can dispose of your compression garments in the usual way. There are no specific disposal instructions.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany