



## JuzoPro Lumbal Xtec / Lumbal Xtec Light

---




Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użyci · Инструкция по эксплуатации · تعليمات الاستخدام · Návod na používanie · Navodila za uporabo

---

### JuzoPro Lumbal Xtec

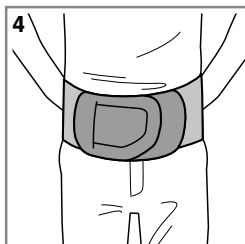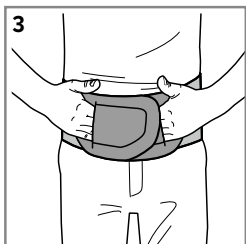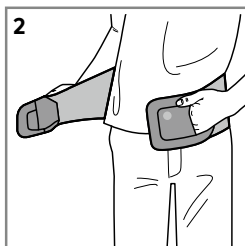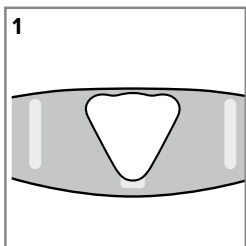### JuzoPro Lumbal Xtec Light

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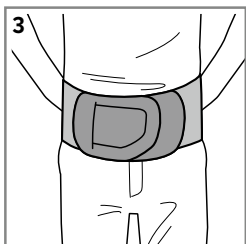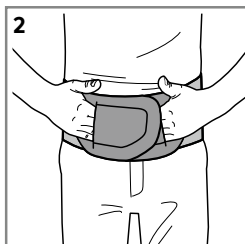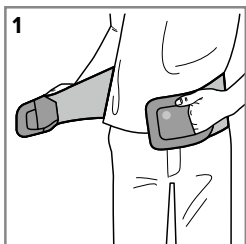## Purpose

Thanks to its compressive knitted fabric, in combination with stabilising rigid elements and optional straps, the JuzoPro Lumbal Xtec / JuzoPro Lumbal Xtec Light orthosis supports the lumbar region of the back, thus relieving it during movement.

JuzoPro Lumbal Xtec: The sacral pad provides stabilisation in the end range of motion and, thanks to friction nubs, has a massaging effect on the muscles.

## Here is how to put on your JuzoPro Lumbal Xtec correctly:

1. Fasten the stabilising girdle as described by your physician or the orthopaedic technician between the two rear bars on the fleece belt and press together firmly. The tip of the stabilising girdle must point downwards (Fig. 1).
2. To make it easier to put on, slip your hands into the finger pockets on the fastener parts. Now place the orthosis around your body (Fig. 2) and close the touch fasteners on the front of the orthosis in the following order: First place the left part of the fastener on your stomach (Fig. 2) and then place the right side of the fastener tightly across it and fasten it (Fig. 3 + 4).

The orthosis is properly in place when it covers the abdomen without exerting too much (painful) pressure. The orthosis must be put on, or must be pulled down, so that the lower edge of the orthosis is situated just above the groin. The orthosis can be readjusted at any time by repeating step 2 above.

## Here is how to put on your JuzoPro Lumbal Xtec Light correctly:

To make it easier to put on, slip your hands into the finger pockets on the fastener parts. Now place the orthosis around your body (Fig. 1) and close the touch fasteners on the front of the orthosis in the following order: First place the left part of the fastener on your stomach (Fig. 1) and then place the right side of the fastener tightly across it and fasten it (Fig. 2 + 3).

The orthosis is properly in place when it covers the abdomen without exerting too much (painful) pressure. The orthosis must be put on, or must be pulled down, so that the lower edge of the orthosis is situated just above the groin.

The orthosis can be readjusted at any time.

## Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product

become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient.

## Washing and care instructions

Please observe the care instructions on the textile label attached to your orthosis. Take the rear stabilising girdle out of the orthosis, close the touch fasteners and wash the orthosis in gentle or delicate cycle at approx. 30 °C. Wash the orthosis separately the first time (possibility of colour bleeding). If washing by hand, rinse the orthosis well and allow it to dry at room temperature. If possible, lay flat. Do not wring it out. We recommend using the mild Juzo Special Detergent. To reduce the drying time, you can place the orthosis on a thick terry towel, roll it up tightly and squeeze it firmly. Do not leave it in the towel and do not place it on a heater or in the sun to dry. Please fasten the stabilising girdle back into the orthosis again when the orthosis is completely dry. This Juzo product must not be cleaned chemically. You can wipe the stabilising girdle clean with a damp cloth. The cover can be washed just like the orthosis

## Materials used

Details are provided on the label sewn into your orthosis.

## Storage information

Store in a dry place and protect from sunlight.

## Indications

### JuzoPro Lumbal Xtec:

Chronic low back pain/ligamentoses (ongoing pain in the lumbar region), degenerative spondylolisthesis (degenerative vertebral slippage), myostatic, muscular insufficiency (postural weakness), osteochondrosis/spondylosis/spondylarthrosis (mild joint wear), post-discectomy syndrome (pain syndrome after surgical removal of intervertebral disc tissue), SI joint syndrome (irritation of the small intervertebral joints)

### JuzoPro Lumbal Xtec Light:

Chronic low back pain/ligamentoses (ongoing pain in the lumbar region), myostatic, muscular insufficiency (postural weakness), symphysis loosening, post-discectomy syndrome (pain syndrome after surgical removal of intervertebral disc tissue), SI joint syndrome (irritation of the small intervertebral joints).

## Contraindications

In case of the following syndromes or clinical pictures, the orthosis should only be worn after prior consultation with the treating physician:

skin diseases or skin irritation;

Wounds in the treatment area are to be covered with sterile dressings.

The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

**Side effects**

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritation) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your symptoms worsen during the time you are wearing the product, please take off the orthosis and consult a medical specialist immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the event of complaints related to this product, such as damage to the material or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

**Disposal**

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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skupina Julius Zorn GmbH ne prevzema nobene odgovornosti.
Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · [juzo.com](http://juzo.com)