





JuzoPro

Lumbal Xtec Strong / Lumbal Xtec Plus







Lumbalorthese

Lumbal orthosis

## JuzoPro Lumbal Xtec Strong

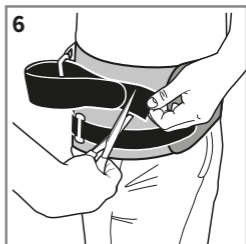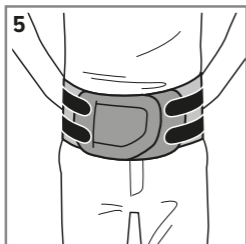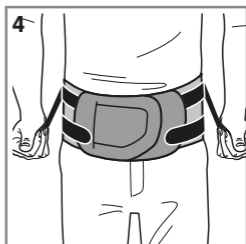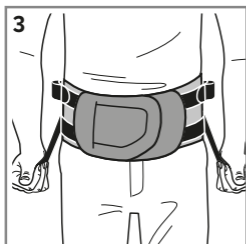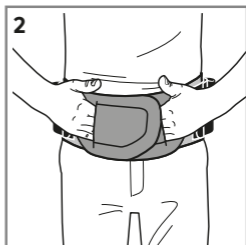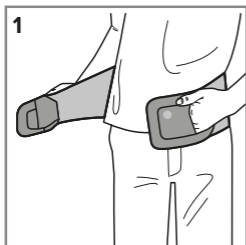## JuzoPro Lumbal Xtec Plus

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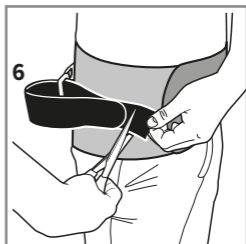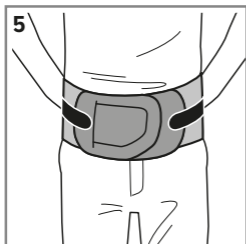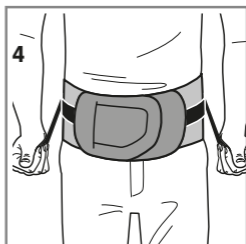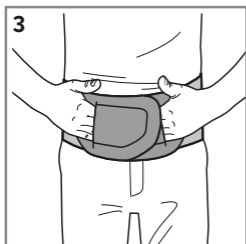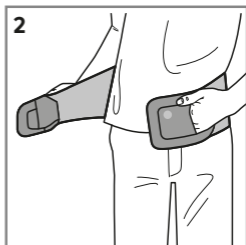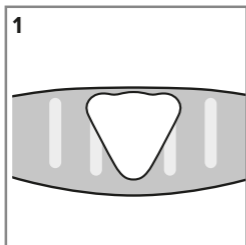## Purpose

Thanks to its compressive knitted fabric, in combination with stabilising rigid elements and optional straps, the JuzoPro Lumbal Xtec Strong / JuzoPro Lumbal Xtec Plus orthosis supports the lumbar region of the back, thus relieving it during movement.

JuzoPro Lumbal Xtec Plus: The sacral pad provides stabilisation in the end range of motion and, thanks to friction nubs, has a massaging effect on the muscles.

## Here's how to put on your JuzoPro Lumbal Xtec Strong correctly:

1. Before applying the orthosis please loosen the pair of straps on each side of the fastening area to ensure the orthosis is free of tension.
2. To make it easier to put on, slip your hands into the finger pockets on the fastener parts. Now place the orthosis around your body (Fig. 1) and close the velcro fastener on the front of the orthosis in the following order: First place the left part of the fastener on your stomach (Fig. 1) and then place the right side of the fastener tightly across it and fasten it (Fig. 2).
3. Now detach the lower two pulling straps and simultaneously pull these away from the body (Fig. 3). You can use these to set the optimum tensioning action of the orthosis for your personal requirements and pain sensation. Next secure the ends of the straps anywhere on the velcro fastening area. Repeat the same steps for the upper two straps (Fig. 4 + 5).
4. You can individually adjust the length of the straps by cutting them above where they attach to the fastening area (Fig. 6).

## Here is how to put on your JuzoPro Lumbal Xtec Plus correctly:

1. Fasten the stabilising girdle as described by your physician or the orthopaedic technician between the two rear bars on the fleece belt and press together firmly. The tip of the stabilising girdle must point downwards (Fig. 1).
2. Before applying the orthosis please loosen the two straps to ensure the orthosis is free of tension.
3. To make it easier to put on, slip your hands into the finger pockets on the fastener parts. Now place the JuzoPro Lumbal Xtec Plus around your body (Fig. 2) and close the touch fasteners on the front of the orthosis in the following order: First place the left part of the fastener on your stomach (Fig. 2) and then place the right side of the fastener tightly across it and fasten it (Fig. 3).

4. Now detach the two pulling straps and simultaneously pull these downwards and away from the body at a slight angle (Fig. 4). You can use these to set the optimum tensioning action of the orthosis for your personal requirements and pain sensation. Next secure the ends of the straps anywhere on the velcro fastening area (Fig. 5).

5. You can individually adjust the length of the straps by cutting them above where they attach to the fastening area (Fig. 6).

The orthosis is properly in place when it covers the abdomen without exerting too much (painful) pressure. The orthosis must be put on, or must be pulled down, so that the lower edge of the orthosis is situated just above the groin. You can re-adjust it at any time if necessary.

## Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient.

## Washing and care instructions

Please observe the care instructions on the textile label attached to your orthosis. Take the rear stabilising girdle out of the orthosis, close the touch fasteners and wash the orthosis in gentle or delicate cycle at approx. 30 °C. Wash the orthosis separately the first time (possibility of colour bleeding). If washing by hand, rinse the orthosis well and allow it to dry at room temperature. If possible, lay flat. Do not wring it out. We recommend using the mild Juzo Special Detergent. To reduce the drying time, you can place the orthosis on a thick terry towel, roll it up tightly and squeeze it firmly. Do not leave it in the towel and do not place it on a heater or in the sun to dry. Please fasten the stabilising girdle back into the orthosis again when the orthosis is completely dry. This Juzo product must not be cleaned chemically. You can wipe the stabilising girdle clean with a damp cloth. The cover can be washed just like the orthosis.

## Material composition

Details are provided on the label sewn into your orthosis.

## Notes for the orthopaedic technician

The stabilisation rods can be adjusted to the individual requirements of the patient.

JuzoPro Lumbal Xtec Strong: To shape the rods or, depending on the indication, remove them completely, simply take them out of the pockets.

JuzoPro Lumbal Xtec Plus: The orthosis can be dismantled simply by detaching the pad and strap element.

## Storage information

Store in a dry place and protect from sunlight.

## Indications

### JuzoPro Lumbal Xtec Strong:

joint wear (facet joint arthrosis), severe pain and irritation in the lumbar region (lumbalgia/lumboischialgia), painful bulging of the discs in the lumbar region (protruded/prolapsed disc), pain in the lumbosacral junction, breaking of the connection between two vertebrae (spondylolysis) guarantee since above-average product quality has always been a tradition at Juzo, degenerative vertebral slippage (degenerative spondylolisthesis/1st degree with lumbalgia), deterioration in posture (myostatic muscular insufficiency/segment instability), sudden onset of pain followed by restriction in movement (lumbago/with sacroiliac joint involvement), constriction of the spinal canal (lumbar spinal canal stenosis)

### JuzoPro Lumbal Xtec Plus:

severe persistent/recurrent pain in the lumbar region (chronic, recurrent lumbalgia/ligamentosis), joint wear (osteocondrosis/spondylosis/spondylarthrosis), mild pain and irritation in the lumbar region (lumboischialgia/radiculopathy), breaking of the connection between two vertebrae (spondylolysis), mild to moderate malformation in the lumbar region (lumbar deformity)

## Contraindications

In case of the following syndromes or clinical pictures, the orthosis should only be worn after prior consultation with the treating physician:

- skin diseases or skin irritations;
- Wounds in the treatment area are to be covered with sterile dressings.

The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

## Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your medical condition deteriorates while the use of our products, please go to your physician immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

## Disposal

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany