
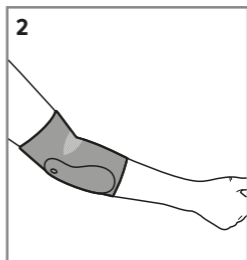
JuzoFlex

Epi Xtra

Ellenbogenbandage

Elbow support

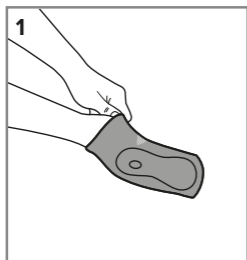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Purpose

The JuzoFlex Epi Xtra guides the affected elbow in an anatomically correct way, relieving strain during movement. Thanks to its bi-elastic knitted fabric in combination with anatomically shaped pads, the support applies compression, therefore promoting lymphatic drainage which results in reduced swelling and pain alleviation. Proprioception and muscular auto-stabilisation of the elbow associated with it are improved.

Here is how to put on your JuzoFlex Epi Xtra correctly:

- 1.** Grab the elbow support at its upper, shorter section and pull the support over the lower arm towards the elbow until the oval, patterned area is centred in the crook of the arm.
- 2.** The epicondyles (prominences on the distal part of the elbow) must be located right in the cut-out of the pressure pads. Please adjust the elbow support as required

The correct fit of your support is guaranteed when it firmly surrounds the elbow without applying too much (potentially painful) pressure on the epicondyles.

Fine adjustment of the elbow support may be made at any time by repeating steps 1 and 2 above.

Please note:

Only wear your support when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the support can only be realized if it is applied correctly. The support is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it be examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the support yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the support is not suitable for treating more than one patient.

Washing and care instructions

Please note the care instructions on the textile label of your support. Wash the support on a gentle or delicate wash cycle at approx. 30 °C. Wash the support separately the first time (colour may run). When hand washing, please rinse the support well and leave it to dry – lying flat if possible – at room temperature. Do not wring it out. We recommend using gentle Juzo special detergent for this. You can reduce the drying time by placing the support on a thick towel, rolling the towel up tightly and firmly pressing out excess moisture. Do not leave it in the towel and do not dry it on a radiator or in the sun. This Juzo product must not be dry-cleaned.

Material used

Details are provided on the label sewn into your wrist support.

Storage information and shelf life

Store in a dry place and protect from sunlight. Supports have a maximum shelf life of 42 months. The usage period of the medical product is printed on the box with an hourglass symbol. Body measurements may change based on the patient's condition and individual circumstances. We therefore recommend that medical specialists check body measurements regularly.

Indications

tennis or golfer's arm (lateral / medial epicondylitis), joint effusion / swelling due to joint degeneration or inflammation (arthritis / arthrosis), sprained elbow (elbow luxation), persistent soft tissue irritation following an injury / operation (chronic / posttraumatic / postoperative), sprains / strains (distortions)

Contraindications

If the following symptoms exist, the support should only be worn after consultation with the physician:

lymph drainage disorders, arterial flow impairment, post-thrombotic conditions; arterial flow disorder, post-traumatic conditions, skin disorders or skin irritations; Wounds in the treatment area are to be covered with sterile dressings.

The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your medical condition deteriorates while the use of our products, please go to your physician immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your support can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Pri neupoštevanju kontraindikacij skupina Julius Zorn GmbH ne prevzema nikakršne odgovornosti.
**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany