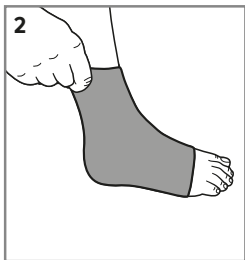
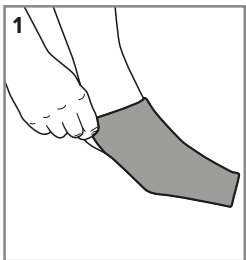
Sprunggelenkbandagen

Ankle supports

Gebrauchsanweisung · Instructions for use · Mode d'emploi
· Istruzioni d'uso · Instruções de utilização · Instrucciones
de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvis-
ning · Kullanım talimatları · Upute za uporabu · Инструкција
по експлуатацији · ارشادات · Navodila za uporabo

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Intended purpose

The JuzoFlex ankle supports provide anatomical guidance for the affected ankle, relieving it during movement. Thanks to their bi-elastic knitted fabric, sometimes in combination with anatomically shaped pads, the supports apply compression, promoting lymphatic drainage which results in reduced swelling and pain alleviation. Proprioception and the resulting muscular auto-stabilisation of the ankle are improved.

Here is how to put on your JuzoFlex Support correctly:

Put on your compression support while you're sitting down to prevent injuries or falling. Rough, unfilled finger nails or sharp-edged rings etc. may damage the fine knitted fabric. Grasp the support on the upper edge and pull it over your ankle (Fig. 1 + 2). The correct fit of your support is guaranteed when it firmly surrounds the ankle without applying too much (potentially painful) pressure on the ankles (malleoli).

Fine adjustment of the support may be made at any time.

Please note:

Only wear your support when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the support can only be guaranteed if it is applied correctly. Although your support is largely resistant to oils, ointments, skin moisture and other environmental influences, the fabric as well as worked in pressure pads can be adversely affected by certain combinations of these factors. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the support yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the support is not suitable for treating more than one patient.

Washing and care instructions

Please note the care instructions on the textile label of your support. Wash the support on a gentle or delicate wash cycle at approx. 30 °C. Wash the support separately the first time (colour may run). When hand washing, please rinse the support well and leave it to dry – lying flat if possible – at room temperature. Do not wring it out. We recommend using gentle Juzo special detergent for this. You can reduce the drying time by placing the support on a thick towel, rolling the towel up tightly and firmly pressing out excess moisture. Do not leave it in the towel and do not dry it on a radiator or in the sun. This Juzo product must not be dry-cleaned.

Material composition

Details are provided on the label sewn into your support.

Storage information and shelf life

Store in a dry place and protect from sunlight. Supports have a maximum shelf life of 42 months. The usage period of the medical product is printed on the box with an hourglass symbol. Body measurements may change based on the patient's condition and individual circumstances. We therefore recommend that medical specialists check body measurements regularly.

Custom-made products can be used immediately. Body measurements may change based on the patient's condition and individual circumstances. Custom-made products are therefore designed for a usage period of 6 months only. After a usage period of 6 months, body measurements must be re-checked by medical specialists.

Indications

Post-operative rehabilitation, degenerative joint disorders, irritation, acute phase after lesions of the ankle ligaments, chronic ankle instabilities, ankle sprain susceptibility or sequelae, quicker resorption of oedemas, effusions and haematomas, post-acute phase after lesions of the ankle ligaments

Contraindications

If the following symptoms exist, the support should only be worn after consultation with the physician: Arterial insufficiency, lymph drainage disorders, post-thrombotic syndromes, skin disorders or skin irritation; open wounds in the treated body area should be kept sterile.

The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritation) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your medical condition deteriorates while the use of our products, please go to your physician immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your support can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Pri neupoštevanju kontraindikacij skupina Julius Zorn GmbH ne prevzema nikakršne odgovornosti.
**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany