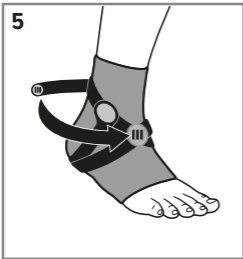
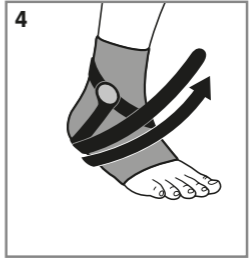
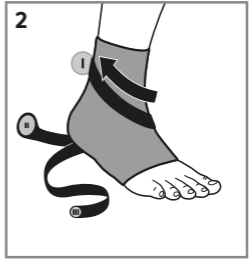
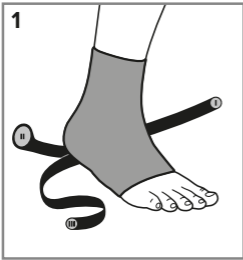
JuzoFlex
Malleo Xtra Strong
Sprunggelenkbandage
Ankle support

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Purpose

The JuzoFlex Malleo Xtra Strong provides anatomical guidance for the affected ankle, thus relieving it during movement. Thanks to its bi-elastic knitted fabric in combination with anatomically shaped pads, the support applies compression, promoting lymphatic drainage which results in reduced swelling and pain relief. Proprioception and the resulting muscular auto-stabilisation of the ankle are improved. The textile reinforcing elements mechanically stabilise the joint.

Here's how to put on your JuzoFlex Malleo

Xtra Strong correctly:

- 1.** Open all the straps and pull it over your ankle so that strap I is located on the inside, and straps II and III on the outside of your foot (Fig. 1).
- 2.** Now run strap I diagonally across your foot arch and attach it to the velcro area provided. Please ensure that your foot remains at a 90° angle to your calf while doing so (Fig. 2).
- 3.** Now attach strap II under tension to strap I on the outside (Fig. 3) and then run strap III diagonally, in the opposite direction to strap I, around the ankle, over strap II and then fasten it onto strap I (Figs. 4 + 5).

The ankle support is fitted correctly if it stabilizes the ankle. Please ensure the straps do not have a constrictive effect and do not exert a too strong (painful) pressure on the ankle.

The straps can be readjusted at any time by repeating steps 2 and 3 above.

We recommend that you follow these instructions closely. Rough, unfiled nails, rings with sharp edges, or incorrect fastening of the straps may damage the fabric.

Please note:

Only wear your support when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the support can only be guaranteed if it is applied correctly. Although your support is largely resistant to oils, ointments, skin moisture and other environmental influences, the fabric as well as worked in pressure pads can be adversely affected by certain combinations of these factors. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the support yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the support is not suitable for treating more than one patient.

Washing and care instructions

Please refer to the washing symbols on the label sewn into your JuzoFlex Malleo Xtra Strong. Always close the velcro fasteners before washing! Wash the support separately the first time (possibility of bleeding). Please do not use any softener! We recommend using the mild Juzo Special Detergent for this.

Material composition

Details are provided on the label sewn into your ankle support.

Storage information and shelf life

Store in a dry place and protect from sunlight. Supports have a maximum shelf life of 42 months. The usage period of the medical product is printed on the box with an hourglass symbol. Body measurements may change based on the patient's condition and individual circumstances. We therefore recommend that medical specialists check body measurements regularly.

Indications

- early functional therapy of distortions of the upper and lower ankle joint
- chronic, post-traumatic or post-operative soft tissue irritation
- injuries of the capsular ligament of the ankle joint
- chronic instability of the ankle joint
- supination prevention
- after a plaster cast

Contraindications

If the following symptoms exist, the support should only be worn after consultation with the physician: lymph drainage disorders, arterial flow impairment, post-traumatic conditions, skin disorders or skin irritations; open wounds in the treated area should be kept sterile.

The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your medical condition deteriorates while the use of our products, please go to your physician immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your support can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Pri neupoštevanju kontraindikacij skupina Julius Zorn GmbH ne prevzema nikakršne odgovornosti.
**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany