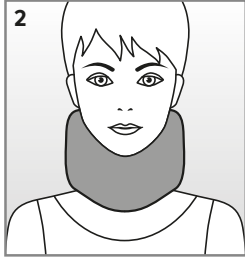
JuzoPro Cervicalstützen JuzoPro Cervical Collar Supports

Gebrauchs- und Pflegeanleitung · Instructions for use and care · Conseils · Avvertenze · Instruções · Pautas · Instructies
Användnings- och underhållsanvisningar
Informationer · Talimatı · Указание · ارشادات

Made in Europe

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Here's how to put on your orthosis

JuzoPro Cervical Collar Support correctly:

Open the orthosis fully and place it around your neck so that the fastener (velcro strap) is located at the back of your neck.

When it is positioned correctly, your chin should be embedded in the opening (recess) provided for it.

The orthosis is especially designed for easy adjustability so you can accommodate the right tightness to help stabilise the neck, and prevents restriction.

You can re-adjust it at any time if necessary.

Purpose

The JuzoPro Cervical Collar Supports stabilise the cervical spine thanks to their contours and potential reinforcing elements.

Please note:

- Only wear your orthosis when prescribed by your physician.
- Medical aids should only be dispensed by suitably trained staff of a medical products supplier.
- The effectiveness of the orthosis can only really be guaranteed if it is applied correctly.
- Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy.
- Storage: Please store in a dry place and at room temperature.
- For hygienic reasons and as your orthosis is an individually tailored product, it is not suitable for use by more than one patient.

Washing and care instructions

Please observe the care instructions on the textile label attached to your orthosis. Wash the orthosis separately the first time (the colour might come out). Close the velcro fastener and wash the orthosis by hand at approx. 30 °C using a laundry detergent for delicate fabrics. We recommend using the mild Juzo Special Detergent for this. Afterwards, please rinse out the orthosis well and allow it to dry at room temperature. To reduce the drying time, you can place the orthosis on a thick terry towel and pat it dry. Do not leave in the towel and do not place on a heater or in the sun to dry. This Juzo product must not be cleaned chemically.

Materials used

The exact details can be found on the textile label on your orthosis.

Indications

- treatment of painful syndromes of the cervical spine
- whiplash trauma
- degenerative changes
- cervical syndrome
- torticollis
- post-traumatic pain syndromes
- osteochondrosis, spondylarthrosis (degenerative disorders of the cervical spine)

Contraindications

If the following symptoms exist, the orthosis should only be worn after consultation with the physician:

- tracheostomy
- hypersensitivity in the neck area
- severe swelling
- skin disorders or skin irritations;
Open wounds in the treated area should be kept sterile.

The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your medical condition deteriorates while using our products, please go to your physician immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

Disposal

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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