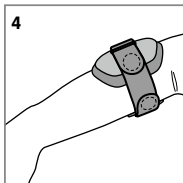
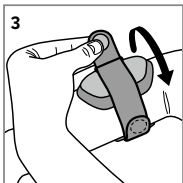
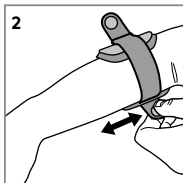
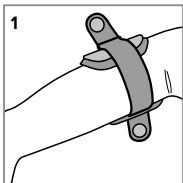
JuzoPro Epi

Gebrauchsanweisung · Instructions for use ·
Mode d'emploi · Istruzioni d'uso · Instruções de
utilização · Instrucciones de uso · Gebruiksaan-
wijzing · Brugsanvisning · Bruksanvisning · Kul-
lanım talimatları · Upute za uporabu · Instrukcja
użytkowania · Инструкция по эксплуатации
· مَادخَت سَالَا تَامِي لَعَت · Navodila za uporabo

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Purpose

Thanks to its shape and cushioning, combined with a pad, the JuzoPro Epi epicondylitis brace relieves the origin of the hand extensor and flexor muscles at the ulnar or radial epicondyle using compression.

Here is how to put on your JuzoPro

Epi correctly:

1. Open both straps and position the brace on the forearm so that the pad on the inside lies directly on the hand extensor or flexor muscles. The position of the pad can be changed to suit individual preference. Its tip, however, must always point towards the elbow. The orthosis should now be surrounding the forearm, opening pointing inwards, about 5 – 6 cm away from the tip of the elbow (Fig. 1).
2. Now position the orthosis by closing the lower strap (Fig. 2).
3. The second strap can now be closed (Fig. 3 + 4).

The brace is positioned correctly if you can hardly feel it when your arm is at rest and obvious counter-pressure is exerted in the forearm as soon as force is applied. You can re-adjust the brace at any time by repeating Steps 2 and 3.

Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult

your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient.

Washing and care instructions

Please follow the care instructions on the textile label on your orthosis. Detach the pressure pad and padding from the orthosis body (velcro fastener) and wipe the body of the orthosis with a moist cloth. The pressure pad and padding should be washed by hand at approx. 30 °C using a laundry detergent for delicate fabrics. We recommend using the mild Juzo Special Detergent for this. Afterwards please rinse all parts well and allow them to dry at room temperature using a clothing rack. To reduce drying time, place the padding in a thick towel before hanging them up, roll them up firmly, and squeeze tightly. Do not leave them rolled up in the towel and do not place on a heater or in the sun to dry. Please do not attach the pressure pad to the orthosis until all the components are fully dry. This product must not be dry cleaned.

Material composition

Details are provided on the label on your orthosis.

Storage information

Store in a dry place and protect from sunlight.

Notes for the orthopaedic technician

The orthosis has already been anatomically pre-formed. Should it be necessary to shape it, this can be performed thermoplastically.

Indications

Lateral epicondylitis (tennis elbow), medial epicondylitis (golfer's/thrower's elbow), tendinopathies in the elbow region (tendon problems), chronic or recurring epicondylitis,

radial nerve palsy (paralysis/wrist drop).

Contraindications

If the following symptoms occur, the orthosis should only be worn after consultation with the physician:

osteochondritis, lymphatic drainage disorders; arterial flow disorder, post-traumatic conditions, skin disorders or skin irritations; Open wounds in the treated area should be kept sterile.

The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative reactions (for example skin irritations) should occur while using this product, please consult your physician or therapist immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your symptoms worsen during the time you are wearing the product, please take off the orthosis and consult a medical specialist immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted material or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Pri neupoštevanju kontraindikacij skupina Julius Zorn GmbH ne prevzema nikakršne odgovornosti.
Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany