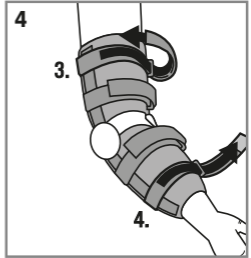
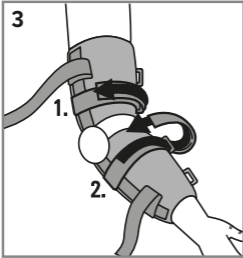
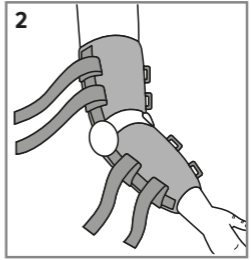
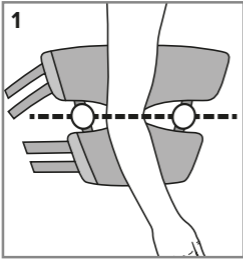
JuzoPro Epi TT

Gebrauchsanweisung · Instructions for use · Mode d'emploi
· Istruzioni d'uso · Instruções de utilização · Instrucciones
de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvis-
ning · Kullanım talimatları · Upute za uporabu · Инструкција
по експлуатацији · ارشادات · Navodila za uporabo

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Intended purpose

The JuzoPro Epi TT stabilises the elbow joint using two rigid, padded guidance splints that are joined together with textile straps and a textile surface in a way that allows for adaptability. These splints are placed on the sides of the joint. The degree of the extension and flexion movement, i.e. how much the elbow can be bent or extended, can be adjusted.

This is how to put on your JuzoPro Epi TT

correctly:

1. Open all the straps, followed by the lengthwise-oriented velcro fasteners on the front, then fold open the orthosis. Now point the wide end of the orthosis, which will enclose your upper arm, in your direction.
2. Position your arm on the orthosis so that the tip of your elbow rests in the space between the upper arm and forearm pads (Fig. 1).
3. Fold the two wings of the orthosis around your arm. The joint pads attached to the hinges of the orthosis should now be located at the same height, and be centered to the left and right of your elbow. Ensure that the centers of the hinge pads are aligned with the center of your elbow joint.
4. Now close the padding-related, lengthwise-oriented velcro fasteners located at the front (Fig. 2).
5. Next close the straps by guiding their ends over your arm, through the plastic buckles and back again. Sequence: Start with the upper arm strap just above the elbow, next the forearm strap just below the elbow, then the higher upper arm strap and finally the lower forearm strap (Figs. 3 + 4).

The orthosis is correctly seated if it firmly encloses the arm without exerting a too strong (painful) pressure and, provided your elbow doesn't need to be immobilized for medical reasons, if your elbow is able to move smoothly.

The orthosis can be readjusted at any time by repeating steps 4 and 5 above.

Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient.

Washing and care instructions

Please observe the care instructions on the textile label of your JuzoPro Epi TT. First open the four straps and the velcro fasteners of the orthosis pads. Now carefully separate the padding from the splints and straps (which are attached by means of velcro fasteners). Ensure the straps and splints do not get detached from one another as you do so. This is very important to ensure that the splints are returned into the correct position in relation to the padding after you have cleaned the orthosis. Next remove the velcro-fastened pads. The pad at the orthosis hinge, and the upper arm and forearm pads should be washed by hand at approx. 30 °C using a laundry detergent for delicate fabrics. Wash the pads separately the first time (the colour might come out). We recommend using the mild Juzo Special Detergent for this. Afterwards, rinse out the pads well and allow them to dry in a free-hanging position at room temperature; do not wring them out. To reduce the drying time of the pads, before hanging them up, place them on a thick terry towel, roll them up tightly and squeeze them firmly. Do not leave in the towel and do not place on a heater or in the sun to dry. This Juzo product must not be cleaned chemically. Do not fasten the splints to the padding again until they are fully dry. To do so, place the splints, with the outer edges of the hinges facing down, in front of you on the floor. The straps should all be under tension to ensure the correct separation of the splints is maintained. Now attach the hinge pads to the velcro pads. Next place the upper arm pad (with the external side down) in the center of the more widely spaced area of the splints so that the tip of the pad is at the same height as the midpoint of the hinge and the remainder of the pad almost touches the hinge pad. Now press the pad into place. Repeat this procedure for the forearm padding in the more closely spaced area of the splints.

Material composition

For specific details please refer to the textile label sewn onto your orthosis.

Storage information

Store in a dry place and protect from sunlight.

Indications

contractures and instabilities in the elbow joint, conservative treatment of elbow luxations with concomitant injuries, juxta-articular pseudoarthrosis as well as posttraumatic joint changes, postoperative care after prosthetic implant, follow-up treatment with stable fractures of the distal upper arm

Contraindications

If the following symptoms exist, the orthosis should only be worn after consultation with the physician: lymph drainage disorders, arterial flow impairment, post-traumatic conditions, skin disorders or skin irritations; open wounds in the treated body region should be covered sterilely.

The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your medical condition deteriorates while using our products, please go to your physician immediately. The manufacturer is not liable for damages/ injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany