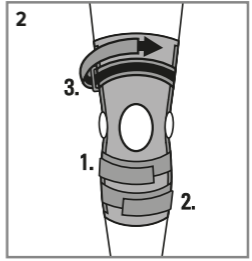
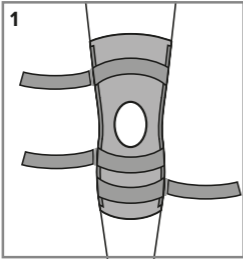
JuzoPro Genu Soft

Gebrauchsanweisung · Instructions for use · Mode d'emploi
· Istruzioni d'uso · Instruções de utilização · Instrucciones
de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvis-
ning · Kullanım talimatları · Upute za uporabu · Инструкция
по эксплуатации · ارشادات · Navodila za uporabo

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Purpose

The JuzoPro Genu Soft stabilises the knee joint using two rigid, padded guidance splints that are joined together with textile straps and a textile surface in a way that allows for adaptability. These splints are placed on the sides of the joint. The degree of the extension and flexion movement, i. e. how much the knee can be bent or extended, can be adjusted.

This is how to put on your JuzoPro Genu Soft correctly:

1. Open all the straps on the front as well as the zipper on the back of the orthosis. Please do not open the touch fasteners of the straps on the back of your orthosis. These determine the final position of the knee joints and should be adjusted by a technician.
2. Now pull the JuzoPro Genu Soft over your foot and up to above your knee until you can see your kneecap in the middle of the circular opening. The zipper must be at the bottom part of the back, the two splints left and right in the middle of your knee joint (Fig. 1).
3. Now close the zipper on the back and then the straps. Sequence: initially, the first strap right under the knee, then the second strap below that and lastly, the thigh strap (Fig. 2).

The correct fit of the orthosis assured when it encloses the leg tightly but does so without exerting a too strong (painful) pressure and your knee can move smoothly, provided it does not need to be secured for medical reasons.

You can re-adjust the orthosis at any time by repeating Step 3.

Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realized if it is applied correctly. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons and as your orthosis is an individually tailored product, it is not suitable for use by more than one patient.

Washing and care instructions

Please note the care instructions on the textile label of your JuzoPro Genu Soft. Close all touch fasteners and the zipper. Now open the strap at the back and the upper parts of the touch fasteners on the splint pockets. The two splints can now be removed from the pockets. Handwash the orthosis fabric with laundry detergent for delicate fabrics at approx. 30 °C. We recommend using gentle Juzo special detergent for this. Wash the orthosis fabric separately the first time (colour may run). Then rinse it well, loosely hang up your JuzoPro Genu Soft and leave it to dry at room temperature. Do not wring it. You can reduce the drying time by placing the orthosis fabric on a thick towel, rolling the towel up tightly and firmly pressing out excess moisture before hanging the orthosis up. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Please re-insert the splints into the pockets only when the orthosis has dried completely. Please note that they must be replaced in exactly the way they were removed. Close the touch fastener at the back only when you next put on the orthosis. Please note that the two splints must be on the left and right in the middle of your knee joint. This Juzo product must not be dry-cleaned.

Materials used

Details are provided on the label sewn into your orthosis.

Storage information

Store in a dry place and protect from sunlight.

Indications

Genu recurvatum based on extension restriction, traumatic or degenerative knee joint changes e. g. osteoarthritis of the knee, mild cruciate ligament instabilities, PCL and ACL, mild collateral ligament insufficiencies and injuries, MCL and LCL, following meniscus injuries or problems, post-operative rehabilitation, prevention during sports, work and everyday activities

Contraindications

If the following symptoms exist, the orthosis should only be worn after consultation with the physician:

pronounced varus or valgus deformity, lymph drainage disorders, arterial flow impairment, post-thrombotic syndromes, all forms of varicosity, ulceration, recent or recurring flebothrombosis, permanent oedema of the lower leg, skin diseases or skin irritations; Wounds in the treatment area are to be covered with sterile dressings, a below-knee compression stocking (AD) should also be worn by elderly patients with a tendency towards swelling and in conjunction with varicosity.

The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your medical condition deteriorates while the use of our products, please go to your physician immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany