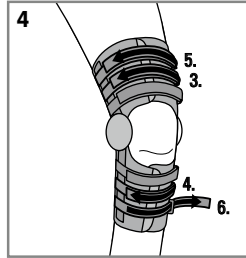
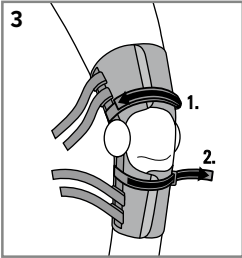
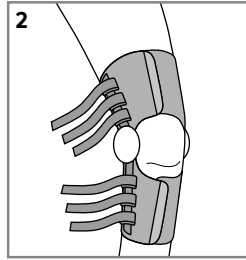
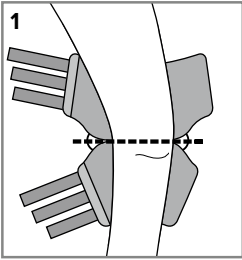
## JuzoPro Genu TT

---

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Kullanım talimatları · Upute za uporabu · Instrukcja użytkowania · Инструкция по эксплуатации · ارشادات · Navodila za uporabo

---

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## Intended purpose

The JuzoPro Genu TT stabilises the knee joint using two rigid, padded guidance splints that are joined together with textile straps and a textile surface in a way that allows for adaptability. These splints are placed on the sides of the joint. With the JuzoPro Genu TT, the degree of the extension and flexion movement, i.e. how much the knee can be bent or extended, can be adjusted.

## How to put on your JuzoPro Genu TT correctly:

1. Open all the straps, followed by the length-wise-oriented velcro fasteners on the front, then fold open the orthosis. You should now point the wide end, which covers the thigh, in your direction.
2. Position your leg within the orthosis so that the hollow of your knee rests in the space between the thigh and calf pads.
3. Fold the two wings of the orthosis around your leg. The joint pads attached to the hinges of the orthosis should now all be located at the same height, and be centered to the left and right of your knee joint. Ensure that the centers of the hinge pads are aligned with the center of your patella.
4. Now close the padding-related, lengthwise-oriented velcro fasteners located at the front.
5. Next close the straps by guiding their ends over your leg, through the plastic buckles and back again. Sequence: guide the first thigh strap over the knee, then the first calf strap below the knee, next the middle thigh strap followed by the middle calf strap, then the uppermost thigh strap and finally the lowermost calf strap.

The orthosis is seated correctly if it firmly encloses the leg without exerting a too strong (painful) pressure and your knee is able to move smoothly (provided your knee does not need to be immobilised for medical reasons).

You can readjust your orthosis any time by repeating steps 4 and 5.

## Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient.

## Washing and care instructions

Please comply with the care instructions on the textile label of your orthosis. Open the straps and the front velcro fasteners and spread the orthosis out in front of you. The pads must be lying completely flat, i.e. there should be no visible upward arching. Next release and remove the two straps that are fastened to the splints by means of velcro fasteners (the first strap above and below the knee respectively). The following points are important to ensure that the splints are returned into the correct position in relation to the padding, as adjusted by the specialist at the store where you bought the orthosis, after you have cleaned the pads. Please pull the end of the uppermost strap gently to the left until the strap buckle on the right side contacts the splint buckle. Now guide the end of the strap over the splint buckle on the left side and back to the right, then attach the velcro fastener onto the strap itself. When the strap is closed in this manner, it should be slightly under tension with no scrunching of the pads. Repeat this procedure for the other three belts. Detach the splints from the padding (velcro fasteners) and remove the disk-shaped pads from the orthosis hinges. To ensure the thigh and calf circumference settings made by the specialist are retained, please do not open the velcro fasteners at the rear of the thigh and calf pads. The joint and orthosis pads should be washed by hand at approx. 30° C using a laundry detergent for delicate fabrics. We recommend using the mild Juzo Special Detergent. Wash the pads separately the first time (possibility of bleeding). Afterwards, rinse out the pads well and allow them to dry in a free-hanging position at room temperature; do not wring them out. To reduce the drying time of the pads, before hanging them up, place them on a thick terry towel, roll them up tightly and squeeze them firmly. Do not leave the orthosis in the towel and do not let it dry on a radiator or in the sun. This product should not be chemically cleaned.

Do not fasten the splints to the padding again until they are fully dry. To do so, place the splints, with the outer edges of the hinges facing down, in front of you on the floor. The straps should all be under tension to ensure the correct separation of the splints is maintained. Now attach the disk-shaped hinge pads to the velcro pads. Center the thigh padding (with the outer edge facing down) on the more widely spaced part of the splints so that the two highest points on the pad almost touch the hinge pad. Now press the pad into place. Repeat this procedure for the calf padding, this time at the more closely spaced part of the splints. Only attach the two loose straps again when applying the orthosis.

## Material composition

For precise details, please see the textile label in your orthosis.

## Storage information

Store in a dry place and protect from sunlight.

## Indications

Gradual post-traumatic mobilisation after knee injuries, postoperative immobilisation, following corrective osteotomies of the lower leg or the thigh, secure knee ligament stabilisation after surgery or acute treatment in case of injuries, postoperative recovery.

## Contraindications

In case of the following syndromes or clinical pictures, the orthosis should only be worn after prior consultation with the treating physician:

lymphatic drainage disorders, arterial circulation problems, post-traumatic conditions, skin diseases or skin irritations; Wounds in the treatment area are to be covered with sterile dressings.

The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

## Side effects

As yet not known. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your doctor or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your doctor before use. If your symptoms worsen during the time you are wearing the product, please take off the orthosis and consult a medical specialist immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

## Disposal

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · [juzo.com](http://juzo.com)