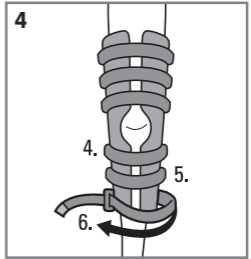
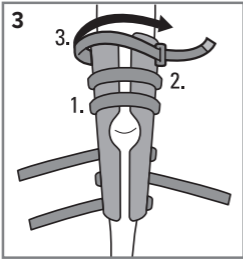
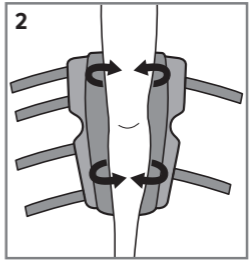
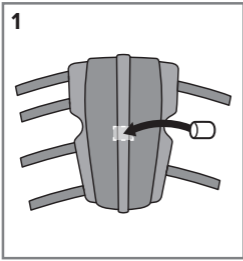
JuzoPro Immo

Gebrauchsanweisung · Instructions for use · Mode d'emploi
· Istruzioni d'uso · Instruções de utilização · Instrucciones
de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvis-
ning · Kullanım talimatları · Upute za uporabu · Инструкция
по эксплуатации · ارشادات · Navodila za uporabo

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Intended purpose

The JuzoPro Immo splints stabilise the knee joint using rigid, padded guidance splints that are joined together with textile straps and a textile surface in a way that allows for adaptability. These splints are placed on the sides of the joint. The degree of the extension and flexion movement, i.e. how much the knee can be bent or extended, can be adjusted.

This is how to put on your JuzoPro Immo correctly:

1. Open all the straps and fold open the orthosis. You should now point the wide end, which covers the thigh, in your direction.
2. Only for JuzoPro Immo 20°: Take the enclosed popliteal pad and attach it, with the lower edge aligned crosswise, to the hook and loop fastener provided in the middle of the orthosis (Fig. 1).
3. Place your leg in the orthosis in such a way that the hollow of your knee rests in the center of or, in the case of JuzoPro Immo 20°, on top of the popliteal pad (Fig. 2).
4. Fold the two padded wings around your leg. Your kneecap should now be visible in the center of the opening provided for it.
5. Now close the straps (sequence: from the knee in the direction of your hip and then from the knee in the direction of your ankle) by passing the straps over the leg, through the plastic buckles and then back again (Figs. 3 + 4).

The correct fit of the JuzoPro Immo is assured when it encloses the leg tightly but does so without exerting a too strong (painful) pressure. The orthosis must immobilize the leg fully.

The JuzoPro Immo can be readjusted at any time by repeating step 5.

Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient.

Washing and care instructions

Please observe the care instructions on the textile label attached to your JuzoPro Immo. Please remove the four aluminum splints from their respective pockets (closed on the thigh side by means of hook and loop fasteners) and close the pockets again. Open all the straps and pull them out of the buckles. Now attach the velcro pad back onto the fleece of the belt to ensure that the orthosis material is not damaged during washing. The orthosis material and popliteal pad (only present for JuzoPro Immo 20°) should be washed by hand at approx. 30 °C using a laundry detergent for delicate fabrics. We recommend using the mild Juzo Special Detergent. Wash the JuzoPro Immo orthosis separately the first time (possibility of bleeding). Afterwards please rinse out the pad material well and allow it to dry in a free-hanging position at room temperature; do not wring it out. To reduce the drying time, before hanging up the pad you can place it on a thick terry towel, roll it up tightly and squeeze it firmly. Do not leave it in the towel and do not place it on a heater or in the sun to dry. Do not insert the aluminum splints back into the pockets and attach the popliteal pad (only for JuzoPro Immo 20°) until the padding material is completely dry. This Juzo product must not be cleaned chemically.

Material composition

Details are provided on the label sewn into your JuzoPro Immo.

Storage information

Store in a dry place and protect from sunlight.

Notes for the orthopaedic technician

Only for JuzoPro Immo 20°: Before adjusting the splints, please insert the supplied popliteal pad in a lateral orientation and centered in the location provided for it (black square fleece). The four stabilizing rods have been anatomically pre-shaped. If necessary (e. g. to make adjustments or when the indication requires) you can individually remove them from the pockets. To do so, open the pockets at the top end of the splints. The pads for the splints are constructed from three parts that are connected by means of hook and loop fasteners. To make adjustments for the leg circumference, open the pad fasteners but leave the splint straps closed while doing so. Now center the middle part of the pad under the patient's leg (only for JuzoPro Immo 20°) and wrap the two wing parts around the top of the leg. To make the JuzoPro Immo easier for the patient to apply and use, it is possible to shorten the splint straps to the actual required length.

Indications

pre- and post-operative immobilization after an PCL rupture / reconstruction or ligament injuries, post-operative immobilization after a meniscus suture, as an alternative to a plaster cast, or after premature removal of a plaster cast, to fully immobilize the joint, rheumatic complaints of the knee, immobilization after a patella luxation, immobilization after strains, or muscular or meniscal injuries

Contraindications

If the following symptoms exist, the orthosis should only be worn after consultation with the physician:

lymph drainage disorders, arterial flow impairment, post-thrombotic syndromes, skin disorders or skin irritations; Wounds in the treatment area are to be covered with sterile dressings.

The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your medical condition deteriorates while the use of our products, please go to your physician immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany