## JuzoPro Malleo

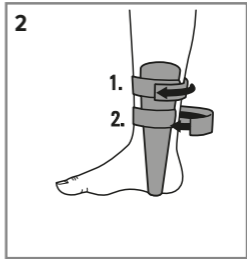
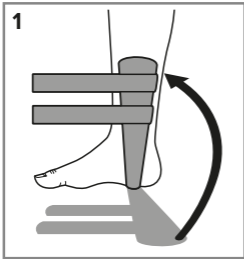
---

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Kullanım talimatları · Upute za uporabu · Инструкция по эксплуатации · ارشادات · Navodila za uporabo

---

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## Intended purpose

Intended the JuzoPro Malleo stabilises the ankle joint with two rigid padded elements that are joined together in a way that allows for flexibility and adaptability and lie on the sides of the joint. It limits the pronation and supination movement, i.e. the inward or outward twisting.

## This is how to put on your JuzoPro Malleo correctly:

1. Open the two straps on the orthosis shells, fold open the shells and lay the orthosis down on the floor in front of you with the pads facing up.
2. Now place your heel on the piece connecting the orthosis shells. As you do this your foot should be at a 90° angle to your lower leg.
3. Now fold both shells around your ankle so that your anklebones rest in the center of the corresponding protrusions provided (Fig. 1).
4. Now pass the orthosis strap around your leg and attach it to the hook and loop pad on the opposite shell. Continue to wrap the strap around your leg and then attach the hook and loop fastener located on the end of the strap to the strap itself.
5. Repeat the previous step for the lower orthosis strap (Fig. 2).

The correct fit of the orthosis is assured when it encloses the ankle tightly but does so without exerting a too strong (painful) pressure.

The orthosis can be readjusted at any time by repeating steps 4 and 5 above.

To achieve an optimal therapy result you should wear the orthosis over your socks while wearing runners/shoes.

## Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient.

## Washing and care instructions

Please observe the care instructions on the textile label on your JuzoPro Malleo. Detach the pads from the orthosis shells (hook and loop fastener) and wipe the shells with a moist cloth. The pads should be washed by hand at approx. 30 °C using a laundry detergent for delicate fabrics. We recommend using the mild Juzo Special Detergent. Afterwards please rinse out the pads well and allow them to dry in a free-hanging position at room temperature. To reduce the drying time, before hanging up the pads you can place them on a thick terry towel, roll them up tightly and squeeze firmly. Do not leave in the towel and do not place on a heater or in the sun to dry. Please do not reattach the pads to the shells until they are fully dry. This Juzo product must not be cleaned chemically.

## Material composition

For specific details please refer to the textile label sewn into your JuzoPro Malleo.

## Storage information

Store in a dry place and protect from sunlight.

## Notes for the orthopaedic technician

JuzoPro Malleo can be adjusted for the particular foot width. To do so detach the pad at the sole of the foot from the shell. You can now loosen the hook and loop fastener and adjust the distance between the shells as required.

## Indications

conservative and post-operative treatment of ligament injuries, chronic instability of the ankle joint, following ligament lesions and / or distortions, stabilization of the ankle during sport

## Contraindications

If the following symptoms exist, the orthosis should only be worn after consultation with the physician: lymph drainage disorders, arterial flow impairment, post-thrombotic conditions skin disorders or skin irritations; open wounds in the treated body region should be covered sterilely.

The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

**Side effects**

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your doctor or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your doctor before use. If your medical condition deteriorates while using our products, please go to your doctor immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

**Disposal**

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany