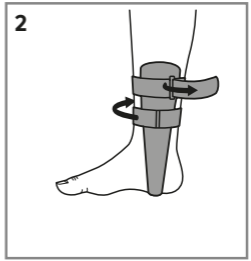
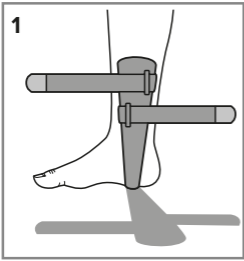
JuzoPro  
Malleo Xtec Light  
Sprunggelenkbandage  
Ankle support

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## Intended purpose

The JuzoPro Malleo Xtec Light stabilises the ankle joint with two rigid padded elements that are joined together in a way that allows for flexibility and adaptability and lie on the sides of the joint. It limits the pronation and supination movement, i.e. the inward or outward twisting.

## This is how to put on your orthosis JuzoPro Malleo Xtec Light correctly:

1. Open the two straps on the orthosis shells, fold open the shells and lay the orthosis down on the floor in front of you.
2. Now place your heel on the piece connecting the orthosis shells. As you do this your foot should be at a 90° angle to your lower leg.
3. Now fold both shells around your ankle so that your anklebones rest in the centre of the corresponding protrusions provided (Fig. 1).
4. Pass the upper orthosis strap through the buckle and around your leg, then fasten it.
5. Repeat the previous step for the lower orthosis strap (Fig. 2).

The correct fit of the orthosis is assured when it encloses the ankle tightly. Please ensure the straps do not have a constrictive effect and do not exert a too strong (painful) pressure.

The orthosis can be readjusted at any time by repeating steps 4 and 5 above. To achieve an optimal therapy result you should wear the orthosis over your socks while wearing runners / shoes.

## Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient.

## Washing and care instructions

Please observe the care instructions on the foot base of the orthosis. Wash the orthosis separately the first time (the colour might come out). For improved cleaning results, we recommend that you remove the straps completely. Wash the orthosis and straps by hand at approx. 30 °C using a laundry detergent for delicate fabrics. We recommend using the mild Juzo Special Detergent for this. Afterwards, please rinse out the orthosis well and allow it to dry at room temperature. To reduce the drying time, you can place the orthosis on a thick terry towel and pat it dry. Do not leave in the towel and do not place on a heater or in the sun to dry. This Juzo product must not be cleaned chemically.

## Material composition

For full details, please refer to the label on the foot base of the orthosis.

## Storage information

Store in a dry place and protect from sunlight.

## Notes for the orthopaedic technician

The orthosis can be adjusted to suit the width of the foot. To do so, release the velcro fastener at the base and adjust the separation between the shells as required.

## Indications

Conservative and post-operative treatment of ligament injuries, chronic instability of the ankle joint, following ligament lesions and / or distortions, stabilisation of the ankle joint during sport

## Contraindications

If the following symptoms exist, the orthosis should only be worn after consultation with the physician: lymph drainage disorders, arterial flow impairment, post-traumatic conditions, skin disorders or skin irritations; Wounds in the treatment area are to be covered with sterile dressings.

The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

**Side effects**

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your medical condition deteriorates while using our products, please go to your physician immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

**Disposal**

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany