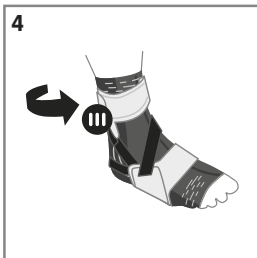
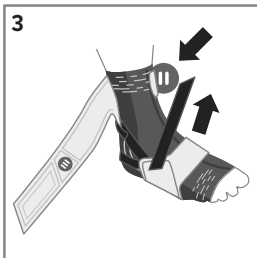
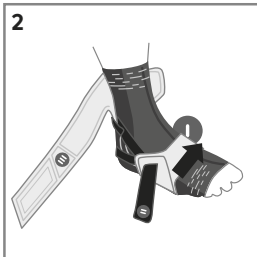
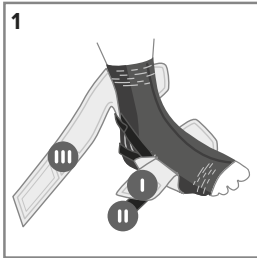
JuzoPro Malleo strong

Gebrauchs- und Pflegeanleitung · Instructions for use and care · Conseils · Avvertenze · Instruções · Pautas · Instructies · Informationer · Talimatı · Указание · ارشادات

Made in Europe

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Here's how to put on your

JuzoPro Malleo strong correctly:

1. Open the Velcro fasteners and then place your heel on the base of the orthosis. Ensure that your foot remains at a 90° angle to your calf while doing so. The black elastic band should be taut and enclose the heel (Fig. 1).
2. Now run strap I horizontally over your forefoot and fasten it (Fig. 2).
3. Next run strap II diagonally to the upper end of the orthosis (Fig. 3).
4. Next run strap III in a circular direction over strap II and fasten it (Fig. 4).

The correct fit of the orthosis is assured when it encloses the ankle tightly but does so without exerting a too strong (painful) pressure. The orthosis can be readjusted at any time by repeating step 2. To achieve an optimal therapy result, you should wear the orthosis on top of the supplied sock and wear a suitable shoe.

Please note:

- Only wear your orthosis when prescribed by your physician.
- Medical aids should only be dispensed by suitably trained staff of a medical products supplier.
- The effectiveness of the orthosis can only really be guaranteed if it is applied correctly.
- Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy.
- Storage: Please store in a dry place and at room temperature.
- As your orthosis is an individually tailored product, it is not suitable for use by more than one patient.

Washing and care instructions

Please observe the care instructions on the inside of the product. Wash the orthosis separately the first time (the colour might come out). Wash the orthosis and straps by hand at approx. 30 °C using a laundry detergent for delicate fabrics. We recommend using the mild Juzo Special Detergent for this. Afterwards, please rinse out the orthosis well and allow it to dry at room temperature. To reduce the drying time, you can place the orthosis on a thick terry towel and pat it dry. Do not leave in the towel and do not place on a heater or in the sun to dry. This Juzo product must not be cleaned chemically.

Materials used

For full details, please refer to the label on the inside of the product.

Disposal

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

Notes for the orthopaedic technician

The integrated stabilizing element has been anatomically preshaped and can be reshaped if necessary. The black lateral strap can be adjusted in length to suit individual requirements.

Indications

- Chronic ankle joint instability
- Mild soft tissue injuries
- Capsular / ligamentous injuries
- Following ligament lesions and / or distortions
- Stabilization of the ankle during sport

Contraindications

If the following symptoms exist, the orthosis should only be worn after consultation with the physician:

- Lymph drainage disorders, arterial flow impairment, post-traumatic conditions
- Skin disorders or skin irritations; open wounds in the treated body region should be covered sterilely

The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your doctor or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your doctor before use. If your medical condition deteriorates while using our products, please go to your doctor immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

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