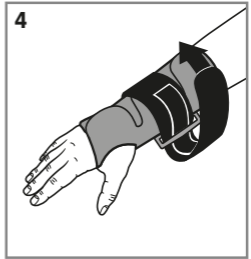
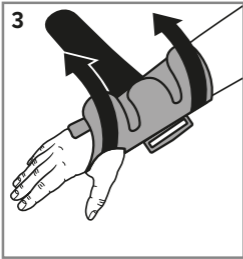
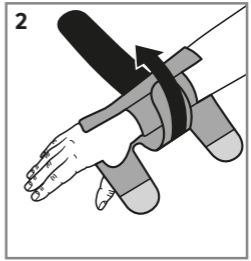
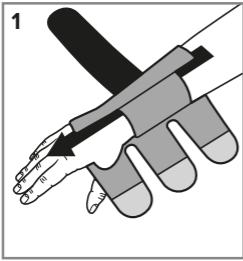
JuzoPro Manu

Gebrauchsanweisung · Instructions for use · Mode d'emploi
· Istruzioni d'uso · Instruções de utilização · Instrucciones
de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvis-
ning · Kullanım talimatları · Upute za uporabu · Инструкция
по эксплуатации · ارشادات · Navodila za uporabo

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Intended purpose

The JuzoPro Manu, made out of inelastic materials with adjustable, partly removable reinforcing elements, immobilises the wrist in at least two movement directions.

This is how to put on your JuzoPro Manu correctly:

- 1.** Open all velcro fasteners and slip your hand into the orthosis so that the velcro fasteners are on top and your thumb protrudes from the opening provided for it.
- 2.** First, fasten the centre velcro, or for the long model fasten the two medium velcros.
- 3.** Then secure the velcro across the hand, and finally secure the fastener on the forearm.
- 4.** Move the circular strap around the wrist and push it through the plastic strap. Then pull it back over it and close the velcro fastener.

Proper fit of your orthosis is ensured if it firmly encloses the wrist without exerting undue (painful) pressure.

You can readjust your orthosis any time by repeating steps 2 – 4.

Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient.

Washing and care instructions

Please observe the care instructions on the textile label attached to your orthosis. Please remove the circular strap. Afterwards remove the three aluminum splints from their pockets. Close all velcro fasteners and wash your orthosis and its strap by hand at a temperature of about 30 °C, using a gentle detergent. We recommend using the mild Juzo Special Detergent for this. Wash the orthosis separately the first time (possibility of colour bleeding). Thereafter, please rinse the orthosis fabric well and let it dry hanging freely; do not wring it out. To reduce the drying time, you can place the orthosis on a thick terry towel, roll it up tightly and squeeze it firmly. Do not leave it in the towel and do not place it on a heater or in the sun to dry. Please return the aluminum splints to their pockets only after the orthosis fabric has dried completely. This Juzo product should not be chemically cleaned.

Material composition

Details are provided on the label sewn into your orthosis.

Storage information

Store in a dry place and protect from sunlight.

Notes for the orthopaedic technician

The three stabilisation splints are anatomically preformed. They may be individually removed from their pockets either for setting, or for specific indications.

Indications

sprains/strains (medium to severe distortions), injuries of joints/tendons (luxations), tendinous synovitis (medium to severe tendovaginitis), paralysis/wrist drop (radial paresis), carpal canal syndrome, chronic irritative conditions (arthrosis and rheumatoid arthritis), plaster cast substitute (posttraumatic/post-operative immobilisation)

Contraindications

If the following symptoms exist, the orthosis should only be worn after consultation with the physician:

lymph drainage disorders, arterial flow impairment, post-traumatic conditions, skin disorders or skin irritations; Wounds in the treatment area are to be covered with sterile dressings.

The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your medical condition deteriorates while the use of our products, please go to your doctor immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Pri neupoštevanju kontraindikacij skupina Julius Zorn GmbH ne prevzema nobene odgovornosti.
**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany