
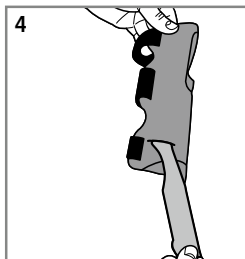
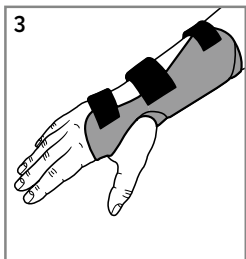
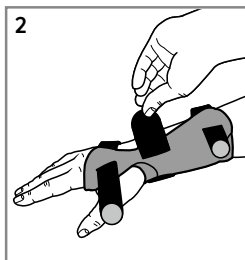
JuzoPro

Manu Xtec Palmar

Handgelenkorthese

Wrist orthosis

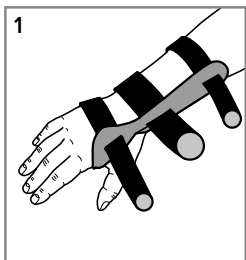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### Purpose

The JuzoPro Manu Xtec Palmar, made out of inelastic materials with adjustable, partly removeable reinforcing elements, immobilises the wrist in two movement directions (ulna / radial and dorsal / volar).

### Here is how to put on your

#### JuzoPro Manu Xtec Palmar correctly:

1. Open all three velcro fasteners on the top and pull the straps through the buckles as far as the hook and loop pad (do not open fully!). Now slip your hand into the orthosis. As you do so, insert your thumb into the opening provided (Fig. 1).
2. First do up the middle velcro fastener, then the strap at the back of the hand, then the fastener on the forearm (Fig. 2).

The orthosis is fitted correctly if it securely encloses the wrist. Please ensure the straps do not have a constrictive effect and do not exert a too strong (painful) pressure on the wrist (Fig. 3).

The orthosis can be readjusted at any time by repeating step 2 above.

The stabilisation splints can be taken out and put back in again via the slit on the side of the pocket (Fig. 4)

#### Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff in a medical products supply store. The effectiveness of the orthosis can only be guaranteed if it is applied correctly. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons and because the orthosis is individually tailored, it is not suitable for treating more than one patient.

#### Washing and care instructions

Please observe the care instructions on the inside of the product. Please remove the stabilising splints before washing, as illustrated in Fig. 4. Wash the orthosis separately the first time (possibility of bleeding). For improved cleaning results, you can remove the straps completely. Wash the orthosis and strap by hand at approx. 30 °C using a laundry detergent for delicate fabrics. We recommend using the mild Juzo Special Detergent for this. Afterwards, please rinse it out well and allow it to dry at room temperature. To reduce the drying time, you can place the orthosis on a thick terry towel and pat it dry. Do not leave it in the towel and do not place it on a heater or in the sun to dry. This Juzo product must not be cleaned chemically.

#### Materials used

For full details, please refer to the label on the inside of the product.

#### Notes for the orthopaedic technician

The integrated stabilising splints have been anatomi-

cally preshaped and can be reshaped if necessary.

### Storage information

Store in a dry place and protect from sunlight.

### Indications

Moderate to severe sprains and strains, tendon/joint injuries, moderate to severe tenosynovitis (inflammation of a tendon and its sheath), carpal tunnel syndrome, rheumatoid arthritis (chronic irritation), osteoarthritis, post-traumatic/postoperative immobilisation (substitute for plaster cast).

### Contraindications

If the following symptoms exist, the orthosis should only be worn after consultation with the physician: lymph drainage disorders, arterial flow impairment, post-traumatic conditions, skin disorders or skin irritations; Wounds in the treatment area are to be covered with sterile dressings.

The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

### Side effects

There are no known side effects when using this product correctly. However, if you notice any adverse changes (e.g. irritation of the skin) during the prescribed use, please consult your doctor or medical retailer immediately. If you know that you are allergic to one or more of the product ingredients, please consult your doctor before using the product. If your symptoms worsen during the time you are wearing the product, please take off the orthosis and consult a medical specialist immediately. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

In the event of complaints related to this product, such as damage to the material or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

### Disposal

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Pri neupoštevanju kontraindikacij skupina Julius Zorn GmbH ne prevzema nobene odgovornosti.