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## ***Dear Customer,***

*The name Juzo stands for a long tradition of high quality and technical expertise.*

### **Here's how to put on your JuzoPro Rhizo correctly:**

- 1** First, slip into the cloth sleeve so that your thumb goes through the opening meant for it (diagram 1).
- 2** Now put the orthosis, with its straps open, over your thumb in such a way that it completely covers the sleeve (diagram 2).
- 3** Fasten the strap round your wrist. If the specialists have also given you thumb-straps, fasten these starting from the bottom upwards and working towards the tip of your thumb (diagram 3).

The JuzoPro Rhizo is in place properly when it is fastened tightly round your wrist and thumb without creating too much (painful) pressure or cutting into you.

You can adjust the orthosis at any time by repeating stage 3.

### **Please note:**

- Only wear your orthosis when prescribed by your physician.
- Medical aids should only be dispensed by suitably trained staff of a medical products supplier.
- The effectiveness of the orthosis can only be realised if it is applied correctly.
- Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy.
- Storage: Please store in a dry place and at room temperature.
- For hygienic reasons and because the orthosis is individually tailored, it is not suitable for treating more than one patient.

*In opting for Juzo, you have selected a product you can rely on.*

EN

### **Washing and care instructions**

Clean the orthosis with a damp cloth.

The cloth sleeves should be washed by hand at 30 °C using a gentle detergent.

We recommend using the mild Juzo Special Detergent for this.

Wash the orthosis separately the first time (the colour might come out).

Then rinse the sleeves thoroughly and hang them out to dry at room temperature.

If you want to shorten the drying time, you can place the sleeves in a thick towel before hanging them up, roll them up firmly, and squeeze tightly.

Do not leave them rolled up in the towel, and do not let them dry on a radiator or in the sun.

This Juzo product must not be dry-cleaned.

### **Indications**

- immobilisation of the radial carpus and also of the saddle joint in the event of rhizoarthritis (saddle joint arthroses)
- arthrosis or arthritis of the radial middle hand and of the MP1
- radial or ulnar (lateral) band lesion (skier's thumb)
- navicular pseud arthrosis in which the grasping function is retained
- Distortions (sprains/strains)
- Dislocations (injuries to tendons/joints)
- Post-traumatic/postoperative immobilisation (alternative to plaster)
- partial immobilisation of the middle hand

### **Contraindications**

If the following symptoms exist, the orthosis should only be worn after consultation with the physician:

- lymph drainage disorders, arterial flow impairment, post-traumatic conditions
- skin disorders or skin irritations;  
Open wounds in the treated body region should be covered sterilely.

The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

### **Side effects**

There are no known side effects when used as directed.

However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately.

If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your medical condition deteriorates while the use of our products, please go to your doctor immediately.

The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

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