
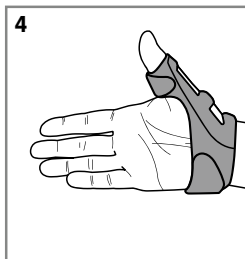
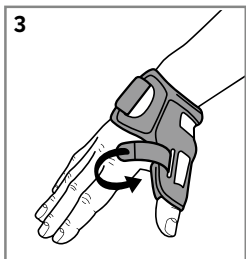
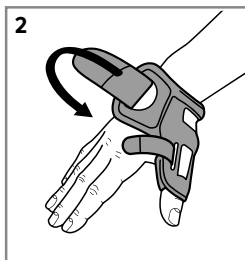
JuzoPro
Rhizo Xtec Soft
Daumenorthese
Thumb orthosis

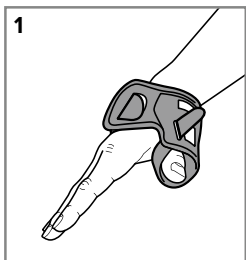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Purpose

The JuzoPro Rhizo Xtec Soft, made of inelastic materials with adjustable reinforcing elements, immobilises the carpometacarpal joint of the thumb, thus relieving it.

Instructions for fitting your JuzoPro Rhizo Xtec Soft correctly:

1. Open both velcro fastenings and pull the tabs through the holes until the velcro fastenings are level with the holes (do not open up completely!).
2. Now place your hand into the orthosis. Your thumb should go through the upper tab. The two cut-outs should be on the outer surface of your thumb (Fig. 1 and 2).
3. First close the lower tab followed by the tab surrounding the thumb (Fig. 3).

The orthosis is fitted correctly if it surrounds the metacarpus and the carpometacarpal (basal) joint of the thumb firmly and without putting too much (painful) pressure on either area (Fig. 4).

You can adjust the orthosis at any time by repeating step 3.

Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient.

Washing and care instructions

Please observe the care instructions on the interior surface of the product. When washing the support for the first time, please wash separately (colour may run). The tabs may be pulled right through the holes in order to allow easier cleaning. Hand wash only in a mild detergent at approx. 30 °C. Please do not use any softener! We recommend using the mild Juzo Special Detergent for this. After washing, rinse the orthosis well and leave it to dry at room temperature. In order to reduce the drying time the orthosis may be placed on a thick terry towel and dabbed dry. Do not bend the stabilising support frame. Do not leave in a towel and do not dry the support on the radiator or in direct sunlight. This Juzo product should not be dry-cleaned.

Material composition

Details are provided on the label sewn into your orthosis.

Notes for orthopaedic technicians

The built-in stabilising frame is made of aluminium and has been anatomically formed into shape. It may, if required, be adjusted and reformed.

Storage information

Store in a dry place and protect from sunlight.

Indications

Immobilisation of the carpometacarpal joint, rhizarthrosis, osteoarthritis of the carpometacarpal joint of the thumb, osteoarthritis or arthritis of the radial metacarpus and of the MP1 joint, radial or ulnar (collateral) ligament lesions (skier's thumb), De Quervain's tenosynovitis (gamer's thumb).

Contra-indications

In the following illnesses the orthosis should only be worn after consulting a physician:

lymph drainage disorders, arterial flow impairment, post-thrombotic conditions, skin diseases or skin irritations; Wounds in the treatment area are to be covered with sterile dressings.

The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative reactions (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your symptoms worsen during the time you are wearing the product, please take off the orthosis and consult a medical specialist immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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