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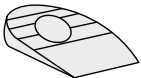
Shoe inserts

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Kullanım talimatları · Upute za uporabu · Instrukcja użycia · Инструкция по эксплуатации · تاداشررا · Navodila za uporabo

1

2

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Intended purpose

Juzo shoe inserts compensate for different leg lengths. They absorb the peak load impact in the heel strike area.

This is how to put on your Juzo Shoe Insert correctly:

Place the shoe insert into your shoe with the writing facing upwards and with the rounded end towards the back of the heel.

1. Juzo Heel Lift Cushion:

Note that you need to insert the heel cushion under the leg that needs raising.

2. Juzo Heel Spur Cushion:

Please note that your heel spur cushions come in a left and right version.

Please note:

Wear your shoe insert only as prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the shoe insert can only be guaranteed if it has been inserted correctly. Should the product become damaged, please consult your specialist supplier. Do not repair the shoe insert yourself – this could compromise its quality and medical efficacy. The shoe insert is intended for use by one patient only.

Washing and care instructions

Wash this product with clear water only and do not use any chemical detergents. Leave to dry fully after washing.

Material composition

100 % silicone

Storage information

Store in a dry place and protect from sunlight. Series products have a maximum shelf life of 36 months. This is followed by a useful life of 6 months.

Indications

Calcaneal spurs, achillodynia, leg length differences, pelvic tilt, impact absorption

Contraindications

The shoe insert should only be used in conjunction with the following conditions after consultation with a physician: skin disorders or skin irritations; Wounds in the treatment area are to be covered with sterile dressings.

The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. Should you be intolerant to the material used in this product, please discuss this with your physician before using the product. If your symptoms worsen during the time you are wearing the product, please take out the shoe inserts and consult a medical specialist immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the material

or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your shoe inserts can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Pri neupoštevanju kontraindikacij skupina Julius Zorn GmbH ne prevzema nobene odgovornosti.
Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany