Juzo Liner

Juzo Liner Silver

Juzo Liner Dual Stretch

Gebrauchsanweisung · Instructions for use · Mode d'emploi
· Istruzioni d'uso · Instruções de utilização · Instrucciones de
uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning
· Kullanım talimatları · Upute za uporabu · Instrukcja użycia ·
Инструкция по эксплуатации · ارشادات · Navodila za uporabo

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Intended purpose

The Juzo Liner, Juzo Liner Silver and Juzo Liner Dual Stretch products are liner stockings for compression therapy and therefore act as accessories for compression products as they support their effect. Wearing liner stockings serves to protect the skin, especially sensitive and stressed skin. They are also used under compression products to secure wound dressings and pads. The Juzo Liner itself does not exert any compression. It is designed to be worn under medical adaptive compression systems only and is also hygienic. The Juzo Liner Silver itself does not exert any compression. It can be used for inflamed skin or chronic skin diseases thanks to the silver thread knitted into it and its antibacterial effect. The Juzo Liner Dual Stretch is an elastic liner stocking with a lightly compressive foot section. It can be combined with medical adaptive compression systems. It is particularly suitable for the treatment and prevention of mild forefoot oedema.

Notes for use

Your specialist retailer will be happy to show you the correct way to use the product. Rough, unfiled finger nails or sharp-edged rings etc. may damage the very fine knitted fabric. We therefore recommend the use of Juzo special gloves. Toe nails and calluses can also cause damage, as can the defective inner lining of shoes. Put your liner stockings on in the morning as soon as you get up. Put on your liner stockings while you're sitting down to prevent injuries or falling.

Juzo Liner / Juzo Liner Silver / Juzo Liner Dual Stretch

1. Using both hands, stretch the stocking and pull it over your foot.
2. Now pull up the stocking evenly and free from any wrinkles – without twisting it or pulling at the edges – and, at the same time, smooth it on to your leg with the flat of your hand. If you have pulled the stocking up too high, simply push it back down to the ankle and then pull it up again with less strength.
3. Once the stocking fits without wrinkles, you can put the compression garment on over the top. Please note the instructions for use for the relevant product.

Please note:

The liner stockings are largely resistant to oils, ointments, skin moisture and other environmental influences. The durability of the knitted fabric may, however, be affected by certain combinations of the above. This is why it is advisable to have a medical retailer check your product regularly. Please contact your specialist retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness. For hygiene reasons, this product is intended for one person only.

Special instructions for Juzo Liner Silver

Juzo Liner Silver has an antibacterial effect, thanks to the knitted-in silver thread. Since silver can conduct heat and electricity, direct contact with electrical sources (e.g. a hair dryer) should be avoided. When Juzo Liner Silver is wet, it should not come into contact with metals (e.g. metal clothes hangers). This may cause discolouration of the silver in the stocking. Tanning agents used in the leather production of the shoe lining may cause discolouration of the stocking and the silver thread in the foot section.

Storage information and shelf life

Store in a dry place and protect from sunlight. Standard products have a maximum shelf life of 36 months. The stockings then have a usage period of 6 months. The usage period depends on correct handling (e.g. care, donning and doffing) and normal use of the products. The usage period of the medical product is printed on the box with an hourglass symbol.

Washing and care instructions

Please note the care instructions on the textile label sewn into your stockings or the printed information. We recommend turning liner stockings inside out before washing and washing them separately the first time (possibility of discolouration). Your liner stockings should be washed every day on a gentle or delicate wash cycle. (Juzo Liner / Juzo Liner Silver: 60°C / Juzo Liner Dual Stretch: 40°C)

Tip: A laundry net provides additional protection for the knitted fabric. Please do not use fabric softener. When hand washing, rinse well and do not wring. We recommend gentle Juzo Special Detergent. You can reduce drying time by placing the stockings on a thick towel after washing, tightly rolling the towel and firmly pressing out excess moisture. Then hang up the stocking loosely. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Please use the delicates setting when drying in a tumble dryer (suitable for tumble-drying: see textile label). Juzo Liner / Juzo Liner Silver / Juzo Liner Dual Stretch are not suitable for dry cleaning.

Special care instructions for Juzo Liner Silver

The stockings can be ironed at a lukewarm temperature without compromising the function of the silver. However, the stockings mustn't be damp during ironing because this may cause discolouration.

Material composition

For precise details, please see the textile label sewn into the product.

Indications – Juzo Liner / Juzo Liner Silver / Juzo Liner Dual Stretch

- To protect the skin
- For fixing wound dressings

Indications – Juzo Liner Silver

- Recurring erysipelas
- Recurring wound infections
- Super-infected dermatitis
- Cumulative toxic contact eczema

Indications – Juzo Liner Dual Stretch

- Preventing forefoot oedema
- Mild forefoot oedema

Absolute contraindications

Advanced peripheral arterial disease (if one of these parameters applies: ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO₂ < 20 mmHg instep). Decompensated heart failure (NYHA III + IV), septic phlebitis, phlegmasia cerulea dolens, acute bacterial, viral or allergic inflammation, swelling of the extremities for unknown reasons.

Relative contraindications

Pronounced weeping dermatoses, intolerance to pressure or of any of the components of the product, severe sensory disorders of the extremities, advanced peripheral neuropathy (e.g. diabetes mellitus), primary chronic polyarthritis, peripheral arterial disease (PAD) stage I/II, malignant lymphoedema.

Important

If there are open wounds in the area to be treated, the affected area must be covered with a sterile wound dressing to ensure appropriate exudate management before Juzo Liner / Juzo Liner Silver / Juzo Liner Dual Stretch can be put on. Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when this product is used correctly. However, if you notice any adverse changes (e.g. irritation of the skin) during the prescribed use, please consult your doctor or medical retailer immediately. If you know that you are intolerant to one or more of this product's components, please consult your doctor before using the product. If your symptoms worsen during the time you are wearing the product, remove the product and please consult your doctor immediately. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

In the event of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration in the patient's health or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

You can dispose of your Juzo Liner / Juzo Liner Silver / Juzo Liner Dual Stretch in the usual way. There are no specific disposal instructions.

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