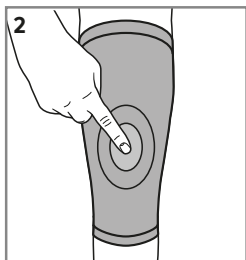
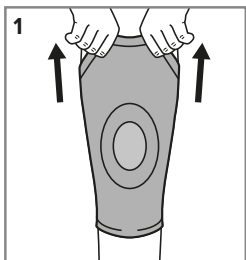
Kniebandagen

Knee supports

Gebrauchsanweisung · Instructions for use · Mode d'emploi
· Istruzioni d'uso · Instruções de utilização · Instrucciones de
uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning
· Kullanım talimatları · Upute za uporabu · Instrukcja użycia ·
Инструкция по эксплуатации · ارشادات · Navodila za uporabo

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Purpose

JuzoFlex knee Supports provide anatomical guidance for the affected knee joint, thus relieving it during movement. Thanks to their bi-elastic knitted fabric, sometimes in combination with anatomically shaped pads, the supports apply compression, promoting lymphatic drainage which results in reduced swelling and pain alleviation. Proprioception and the resulting muscular auto-stabilisation of the knee joint are improved.

Here is how to put on your Juzo Knee Support correctly:

1. Pull the knee support over your forefoot up to the right place on your knee.
2. Then position the knee support so that the patella (kneecap) is enclosed by the patella pad.

Put on your support while sitting down to prevent injuries or falling. The support fits properly when it surrounds the knee without applying too much (painful) pressure. You can re-adjust the support by repeating step 2 at any time. Caution: rough, unfiled finger nails or sharp-edged rings etc. may damage the knitted fabric. Toe nails and calluses can cause damage, as can the defective inner lining of shoes.

Please note:

Wear your knee support only per your physician's prescription. Knee supports should only be dispensed by trained personnel at a medical specialty store. As therapy progresses or new symptoms appear, it will be necessary to take new measurements and possibly to produce a new knee support. Although your knee support is largely resistant to oils, ointments, skin moisture and other environmental influences, the fabric as well as a possibly worked in pressure pad can be adversely affected by certain combinations of these factors. It is therefore recommended to have your knee support checked regularly by your medical specialist retailer. If your knee support becomes damaged please consult your specialty retailer. Do not repair it yourself as this could impair the quality and effectiveness. For hygienic reasons, the support is not suitable for treating more than one patient.

Washing and care instructions

Please note the care instructions on the textile label sewn into your stockings or the printed information. Do up the fasteners (if there are any) before washing! Wash the supports separately the first time (colour may run). Your support should be washed every day on a gentle or delicate wash cycle. Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener! When hand washing, rinse well and do not wring. We recommend the gentle Juzo special detergent. You can reduce drying time by placing the support on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang the support up loosely. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Juzo supports are not suitable for dry cleaning.

Material composition

Details are provided on the label sewn into your knee support.

Storage information and shelf life

Store in a dry place and protect from sunlight. Supports have a maximum shelf life of 42 months. The usage period of the medical product is printed on the box with an hourglass symbol. Body measurements may change based on the patient's condition and individual circumstances. We therefore recommend that medical specialists check body measurements regularly.

Indications

Arthrosis in femoro-patellar joint, Following surgery, Knee joint distortions and effusions, Following fractures, Post-traumatic oedema, Contusions, To aid mobilization after prolonged immobility, Sprains, Subluxation and dislocation of the patella, Meniscopathy

Contraindications

If the following symptoms exist, the support should only be worn after consultation with the physician: Arterial insufficiency, all forms of varicosity, following thrombosis, ulceration, recent or recurring flebothrombosis, permanent oedema of the lower leg. Skin diseases or skin irritations; Wounds in the treatment area are to be covered with sterile dressings. A below-knee compression stocking (AD) should also be worn by elderly patients with a tendency towards swelling and in conjunction with varicosity.

The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician of your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your symptoms worsen during the time you are wearing the product, please take off the support and consult a medical specialist immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your support can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · juzo.com ·