# Juzo SoftCompress

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## Intended purpose

The Juzo SoftCompress bandaging aids are products for compression therapy. Compression therapy reduces oedema and accelerates venous return. The bandaging aids, which can be worn under short-stretch bandages or medical adjustable compression systems (Juzo ACS Light and Juzo Compression Wrap), are skin-friendly and provide additional compression, cushioning and ideal pressure distribution.

## Notes for use

Your specialist dealer or your therapist will be happy to show you how to apply the products correctly. The standard products in the Juzo SoftCompress bandaging aid range have some green seams which can be used to shorten the length of the bandaging aid. Please note that the products must be washed before you shorten them due to the natural shrink properties of the cotton.

## Please note

Only wear compression garments if they have been prescribed by a doctor. The products should only be dispensed by properly trained staff at a medical retailer.

Progress in the success of the treatment or a change in condition will require re-measurement and a new product may be needed.

Please contact your specialist retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness.

## Washing and care instructions

Wash the Juzo SoftCompress after each use. Please note the care symbols on the textile label sewn into the product. All touch fasteners must be closed prior to washing. The products can be washed in the machine at 60 °C using a laundry detergent for delicate fabrics. Tip: a laundry net additionally protects the product. Please do not use fabric softener. Never use bleach. Stretch the wet bandage before drying it. You can reduce drying time by placing the Juzo SoftCompress on a thick towel, rolling the towel up tightly and firmly pressing out excess moisture. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Juzo SoftCompress can be tumble-dried but must not be ironed.

## Treatment and disinfection

(for hospital use)

This product has been successfully tested in accordance with DIN ISO TS 15883-5, SOD 19-002, EN ISO 17664, SOD 19-01 and the Robert Koch Institute "Hygiene Requirements for the Reprocessing of Medical Devices" guideline. The Juzo SoftCompress can be quantifiably treated, cleaned and disinfected in compliance with the provisions of the guideline compiled by the German Society for Hospital Hygiene (DGKH) and the German Society for Sterile Supply (DGSV).

## Material composition

Top cover: 100% cotton

Foam: 100% polyurethane

## Storage information and shelf life

Store in a dry place and protect from sunlight. Standard products have a maximum shelf life of 36 months. The products then have a usage period of 6 months. The usage period depends on correct handling (e.g. care, donning and doffing) and normal use of the products. The usage period of the medical product is printed on the adhesive label on the packaging with an hourglass symbol. Custom-made items are for immediate use and, for this reason, do not have a usage period date.

## Indications

- Lymphoedema
- Phleboedema
- Lipoedema
- Fibrosis
- Post-traumatic oedema
- Post-operative oedema

### Absolute contraindications

- Advanced peripheral arterial disease (if one of these parameters applies: ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO<sub>2</sub> < 20 mmHg instep). When inelastic materials are used, compression garments can still be worn with an ankle artery pressure between 50 and 60 mmHg under continual clinical supervision.
- Decompensated cardiac insufficiency (NYHA III + IV)
- Septic phlebitis
- Phlegmasia coerulea dolens
- Acute bacterial, viral or allergic inflammation
- Swelling of the extremity of unknown cause

### Relative contraindications

- Pronounced weeping dermatoses
- Incompatibility with pressure or product ingredients
- Severe sensitivity disorders in the extremities
- Advanced peripheral neuropathy (e.g. in cases of diabetes mellitus)
- Primary chronic polyarthritis
- Peripheral artery disease (PAD) stage I/II
- Malignant lymphoedema

### Important

If there are open wounds in the area to be treated, the affected area must be covered with an appropriate wound dressing or bandage before the Juzo SoftCompress may be used.

Softcompress cannot accept liability for any failure to comply with these contraindications.

### Side effects

There are no known side effects when this product is used correctly. However, if you notice any adverse changes (e.g. skin irritation) during the prescribed use, please consult your doctor, therapist or medical retailer immediately. If you know you have an intolerance of any component of this product, please consult your doctor or therapist before using it. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

In the event of complaints related to this product, such as damage to the material or a problem with the fit, please contact your medical retailer directly. Only serious incidents that may lead to a major deterioration in the patient's health or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) of Regulation (EU) 2017/745 (MDR).

### Disposal

You can dispose of your Juzo SoftCompress in the usual way. There are no specific disposal instructions.

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**Vertrieb: Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany