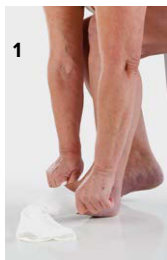
Juzo Ulcer Pro

Juzo Ulcer Pro Add

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Kullanım talimatları · Upute za uporabu · Инструкция по эксплуатации · ارشادات · Navodila za uporabo

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Intended purpose

The Juzo Ulcer Pro treatment set is a medical compression stocking system for the treatment of crural ulcers (Ulcer cruris) and contains two compression stockings that are used in combination with one another. The Juzo Ulcer Pro treatment set comprises a beige stocking (Juzo Ulcer Expert) and two white liner stockings (Juzo Ulcer Liner) that are used in alternation* and serve to hold in place the wound dressing or bandage. When putting on the Juzo Ulcer Liner, make sure there are no creases in it and the bandage or wound dressing doesn't slip. These two stocking components may be used as follows: The liner stocking can be worn day and night on the affected leg for continuous compression. The beige stocking is worn on top of the liner stocking during the day to increase the level of compression.

Notes for use

Put on your compression garments in the morning straight after you get out of bed. Put on and put of your compression garment while sitting down to prevent injuries or falling. Rough, unfiled finger nails or sharp-edged rings etc. may damage the very fine knitted fabric. We therefore recommend the use of Juzo special gloves. Toe nails and calluses can cause damage, as can defective inner lining of shoes.

Putting on the stockings (figs. 1 – 8)

1. To put on the white liner stocking, roll the stocking over your forearm and continue to grasp the foot area with your fingers from the inside while doing so. Pull the stocking off your forearm to turn it inside out down to the heel. Now use both hands to stretch the foot opening and slip the foot portion over the foot to the instep. Make sure the knitted heel area is located on your heel.
2. Gradually slip the white liner stocking over your foot – thereby turning it the right way round again.
3. Now pull up the stocking evenly and creasefree – without twisting it or pulling on the top band – and use the flat of your hand to shape the stocking to your leg. If you pull the stocking up too far, simply slide it back down to your ankle and slip it up your leg again with less tension this time.
4. Now pull the enclosed Juzo Slippie Donning Aid over the foot until the shorter tab is located over your instep and the longer tab is located on the sole of your foot.
5. Next pull the beige stocking over the Juzo Slippie Donning Aid until the knitted heel area is located on your heel. While doing so, make sure that the end of the donning aid continues to extend out the front of the stocking opening in the toe area.
6. Next slide the stocking upwards in sections towards the knee while using a flat hand to shape it to your leg. Please note that the best wearing comfort is obtained when the top band of the beige stocking is located approx. 2 cm below the white liner stocking.
7. Finally, remove the Juzo Slippie donning aid by pulling it

forwards out of the foot portion.

Taking off the stockings (figs. 9 – 10)

Do not take both stockings off at the same time, instead first fold down the beige stocking, then pull it down over the calf and carefully over the heel, instep and foot. Once you have taken off the beige stocking completely, repeat these steps to take off the white liner stocking.

Please note:

Only wear compression garments if they have been prescribed by a doctor. Compression garments should only be dispensed by properly trained staff at a medical retailer. Your Juzo compression garments are largely resistant to oils, ointments, skin moisture and other environmental influences. The durability of the knitted fabric may be affected by certain combinations of the above. This is why it is advisable to have your compression garments checked regularly at a medical retailer. Please contact your specialist retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness. For hygiene reasons, this product is intended for one person only.

Washing and care instructions

Please note the care instructions on the textile label sewn into your compression garment. We recommend turning compression garments inside out before washing. Wash compression garments separately when washing for the first time (they may lose colour). If possible, your compression garments should be washed every day on a gentle or delicate wash cycle (Juzo Ulcer Expert: 40 °C, Juzo Ulcer Liner: 60 °C). Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener! When hand washing, rinse well and do not wring. We recommend the gentle Juzo special detergent. You can reduce drying time by placing the compression garment on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang up the compression garment loosely. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Please use the delicates setting when drying the Juzo Ulcer Liner in the tumble dryer. The Juzo Ulcer Expert is **not** suitable for tumble drying. Juzo compression garments are not suitable for dry cleaning.

Material composition

For precise details, please see the textile label sewn into your compression garment.

Storage information and shelf life

Store in a dry place and protect from sunlight. Standard products have a maximum shelf life of 36 months. The compression garment then has a usage period of 6 months. The usage period depends on correct handling (e.g. care, donning and doffing) and normal use of the products. The usage period of the medical product is printed on the box with an hourglass symbol. Custom-made products are for immediate use. Body measurements may change based on the patient's condition and individual

circumstances. Custom-made products are therefore designed for a usage period of 6 months. After the 6-month usage period, body measurements must be re-checked by a medical specialist. In the event of significant changes in measurements of the body part that is being treated (e.g. based on therapeutic success, weight change), re-measurement and supply of a new product may be required earlier, before the 6-month usage period comes to an end.

Indications

Treatment of *Ulcus cruris venosum* type ulcers as an alternative or complement to compression bandages following oedema reduction, prevention of recurrent *Ulcera cruris venosa* (recidive prophylaxis).

Absolute contraindications

Advanced peripheral arterial disease (if one of these parameters applies: ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO₂ < 20 mmHg instep).

Decompensated heart failure (NYHA III + IV), septic phlebitis, phlegmasia cerulea dolens, acute bacterial, viral or allergic inflammation, swelling of the extremities for unknown reasons.

Relative contraindications

Pronounced weeping dermatoses, intolerance to pressure or of any of the components of the product, severe sensory disorders of the extremities, advanced peripheral neuropathy (e.g. diabetes mellitus), primary chronic polyarthritis, peripheral arterial disease (PAD) stage I/II, malignant lymphoedema.

Important

If there are open wounds in the area to be treated, the affected area must be covered with an appropriate wound dressing or bandage before the compression garment can be applied. Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when used as directed. However, if negative reactions (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your doctor or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your doctor before use. If your medical condition deteriorates while using our products, please go to your doctor immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2

(65) Regulation (EU) 2017/745 (MDR).

Disposal

You can dispose of your compression garments in the usual way. There are no specific disposal instructions.

* The Juzo Ulcer Pro Add set (replacement set) contains only the white liner stockings and is available in medical supply stores.

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