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## Notes for application

Your specialist dealer or therapist will show you how to use it correctly.

- Wear your compression garments for 23 hours non-stop (or as advised by your doctor).
- Compression garments are perfect for whole body treatments and they offer a certified UPF 80 UV protection factor in accordance with UV Standard 801. The UV protection only applies to those body regions that are covered with compression garments.
- Rough, unfiled finger nails or sharp-edged rings etc. may damage the very fine knitted fabric. Therefore please use Juzo Special Gloves.
- For compression garments with a silicone border, please note: The relevant part of the body should be depilated and free from oils, powder, creams and ointments.

## Purpose

The consistent application of uniform surface pressure by the compression garments has a beneficial effect on scar maturation and development. Therapeutic application of compression garments results in decreased blood circulation and increased tissue elasticity. The aim is to reduce the risk of the formation of hypertrophic scars and/or keloids or achieve a reduction in their appearance. The objective of compression therapy is to flatten the scar, increase tissue elasticity and reduce scar rigidity.

## Stump shrinker leg

1. Turn the shrinker inside out, except for its end part.
2. Take the shrinker, folded over itself, in both hands, and slip your stump into it.
3. Now pull up the shrinker evenly and free of wrinkles – without twisting it or pulling on the top band – and use the flat of your hand to shape it to your leg. If you pull the shrinker up too far, simply slide it back down and slip it up your leg again with less tension this time. Tip: Fold over the silicone border (if present) before putting on the shrinker.

## Stump shrinker with knitted edges or silicone borders

4. Starting from the top, down to half way, turn the shrinker inside out.
5. Pull the shrinker over your hand and work your way up your arm to the elbow.
6. Work the folded section of the shrinker over your elbow and pull it step by step towards your shoulder. Spread out the shrinker's knitted fabric so that there are no creases. When the shrinker is in the right position and fits perfectly, it will neither cause uncomfortable pressure or pinching, nor restrict your movements. Tip: Before donning, fold over the silicone border (if applicable).

## Stump shrinker with shoulder cap and strap or bra fastening

7. Place one part of the shoulder strap across your chest and the other over your back. Next, fasten both ends together. Please bear in mind that you can change the length of the shoulder strap using the adjuster if it is too loose or too tight.

## Please note:

- Only wear compression garments if they have been prescribed by a doctor.
- Compression garments should only be dispensed by properly trained staff at a medical retailer.
- Progressing the success of treatment or a change in condition means that the compression garments will have to be remeasured and, if necessary, new items will need to be made. This is why it is advisable to have your compression garments checked regularly at a medical retailer.
- Please contact your specialist retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness.
- For hygiene reasons, this product is intended for one person only.

## Washing and care instructions

Please note the care instructions on the textile label sewn into your compression garment. Do up any fasteners before washing! We recommend turning compression stockings with silicone borders inside out before washing. Wash compression garments separately when washing for the first time (they may lose colour). Your compression garments should be washed every day on a gentle or delicate wash cycle (40 °C). Tip: A laundry net provides additional protection for the knitted fabric. Please do not use fabric softener. When hand washing, rinse well and do not wring. We recommend the gentle Juzo Special Detergent. You can reduce drying time by placing the compression garment on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang up the compression garment loosely. Do not leave it in the towel and do not dry on a radiator or in the sun. Compression garments are not suitable for dry cleaning or tumble drying.

## Material composition

For precise details, please see the textile label that is sewn into your compression garment.

## Storage information

- Store at room temperature in a dry place and protect from direct sunlight.
- Standard products have a maximum shelf life of 36 months. The compression garment then has a maximum wearing period of 6 months. The shelf life is printed on the box with an hourglass symbol.
- Custom-made items are for immediate use and do not have a shelf life.
- In terms of medical effectiveness (compression effect), compression garments are generally intended to be worn for 6 months. This depends on the right handling (e.g. care, donning and doffing) and normal use of the products.

## Indications

- damage to the skin and healing for skin with hypertrophic scars and keloids
- after injuries / burns / operations
- to improve the scar / wound healing through to completely healing the scar
- to stabilise the surgical outcome

## Contraindications

### Absolute contraindications

- advanced peripheral arterial occlusive disease
- decompensated cardiac insufficiency
- septic phlebitis
- phlegmasia coerulea dolens

### Relative contraindications

- pronounced weeping dermatoses
- intolerance to compression material
- severe sensitivity disorders in extremities
- advanced peripheral neuropathy (e.g. diabetes mellitus)
- primary chronic polyarthritis
- skin diseases or skin irritations;  
Wounds in the treatment area are to be covered with sterile dressings.

For chronic lymphoedema or lipoedema, we recommend our lymphological products.

Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

## Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your doctor or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your doctor before use. The manufacturer and any of its affiliates is not liable for damages / injuries which have been caused by improper handling or misuse.

## Disposal

You can dispose of your compression garments in the usual way. There are no specific disposal instructions.

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**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany