
JuzoPro

Patella Xtec Plus







Knie-Softorthese

Knee soft orthosis


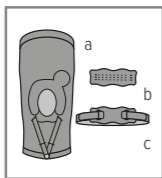

### JuzoPro Patella Xtec Plus
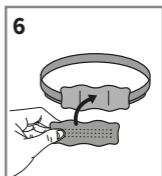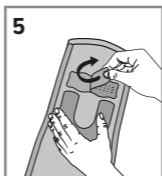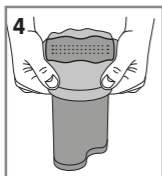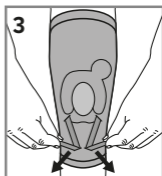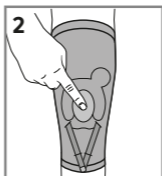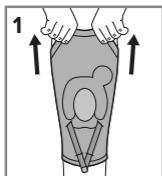
### JuzoFlex Patella Xtra

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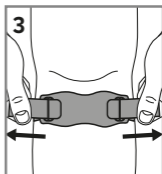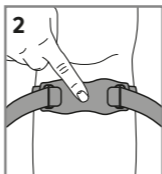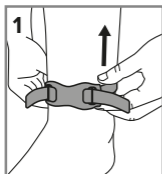## Intended purpose

The textile compressive JuzoPro Patella Xtec Plus knee orthosis gives the patella gentle guidance using a silicone pressure pad. Additional adjustable tensioning elements are intended to exert a positive influence on the path of the patella.

Individual parts (components) of the JuzoPro Patella Xtec Plus:

- Knee support (a)
- Guidance pad (b)
- Patellar tendon strap (c)

## This is how to put on your JuzoPro Patella Xtec Plus correctly:

1. Pull the knee orthosis over your forefoot up to the right place on your knee.
2. Then position the knee orthosis so that the patella (kneecap) is enclosed by the patella pad.
3. You can strengthen the effect on the patella by individually adjusting the strap fastening. Please do this when bending the knee slightly.

The orthosis fits properly when it surrounds the knee without applying too much (painful) pressure.

You can re-adjust the orthosis by repeating step 3 at any time.

## Modification of the JuzoPro Patella Xtec Plus to suit treatment

(only in consultation with your doctor)

4. To dismantle the JuzoPro Patella Xtec Plus, you have to turn the knee orthosis inside out (reverse it).
5. Release the guidance pad from the touch fastener.
6. Then fasten the guidance pad to the patellar tendon strap so that the outside edges line up.

## How to put on your JuzoFlex Patella Xtra correctly:

1. Pull the patellar tendon strap over your forefoot and ankle and up to the right place. Alternatively, you can open up one side of the touch fastener completely and put on the strap as described in step 2 (positioning).
2. You can then position the patellar tendon strap directly below the patella (kneecap).
3. You can adjust the patellar tendon strap by simply pulling it with the help of the cords.

The correct fit of your support is guaranteed when it firmly surrounds the knee without applying too much (painful) pressure.

## Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient.

## Washing and care instructions

Please note the care instructions on the textile label on your orthosis. Remove the patellar strap, close the touch fastener and wash the orthosis on a gentle or delicate wash cycle at approx. 30°C. Wash the orthosis separately when washing for the first time (it may lose colour). When hand washing, please rinse the orthosis well and leave it to dry - lying down if possible - at room temperature. Do not wring it out. We recommend using the gentle Juzo special detergent for this. You can reduce the drying time by placing the orthosis on a thick towel, rolling the towel up tightly and firmly pressing out excess moisture. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Please do not fasten the patellar strap to the orthosis again until the orthosis is completely dry. This product is not suitable for dry cleaning. You can wash the patellar strap by hand.

## Material composition

For precise details, please see the textile label that is sewn into your orthosis.

## Storage information

Store in a dry place and protect from sunlight.

## Indications

- Patellofemoral pain syndrome in cases of muscular imbalance
- Patellar maltracking
- Patellar lateralisation
- Hypermobility patella
- Misalignment
- Patella tip syndrome (jumper's knee)
- Osgood-Schlatter disease
- Following patellar tendon injuries
- Second phase after patellar stabilisation surgery
- High-riding patella (patella alta)

## Contraindications

Only in consultation with your doctor, the orthosis should be worn for the following conditions:

- Arterial circulation problems, all forms of varicose veins, post-thrombotic conditions, ulcerations, recently suffered or recurring thrombophlebitis, permanent below-knee oedemas.
- Skin disorders or irritations; open wounds in the treatment area are to be covered with sterile dressings.
- A compression stocking up to the knee (AD) is also to be worn by people with varicose veins or older people susceptible to swelling.

Liability for failure to comply with the contraindications cannot be accepted by the Julius Zorn GmbH Group.

## Side effects

There are no known side effects when using this product correctly. However, if you notice any adverse changes (e.g. irritation of the skin) during the prescribed use, please consult your doctor or medical retailer immediately. If you know that you are allergic to one or more of the product ingredients, please consult your doctor before using the product. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

## Disposal

Your orthosis can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Pri neupoštevanju kontraindikacij skupina Julius Zorn GmbH ne prevzema nikakršne odgovornosti.
**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany