Adhäsions-Lotion

Alkoholfrei

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Adhesive Lotion

Alcohol-free

For medical compression stockings and supports

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EN

Adhesive Lotion

Intended purpose: special roll-on for the comfortable fastening of compression stockings, knee supports etc.
Application: Apply the adhesive to the skin, put the stocking in the desired position and press briefly. The stocking stays in position the entire day without additional support. After use, put the cap tightly back on the adhesive lotion. This allows the adhesive action to last as long as the contents. The adhesive lotion washes off with clear water and doesn't leave any spots behind.
Storage information, shelf life, disposal: the shelf life of the product is written on the bottle. Do not use after the expiry date. Only store the product sealed in the original bottle and do not use products in damaged packaging. Keep the container tightly sealed and upright. Protect from frost, heat and sunlight and store in a dry place. Keep this product away from children and people with mental disabilities. Dispose of this material and its container at hazardous or special waste collection point.

Usage limitation: always read the product information before use. The product is only for personal use. It is not advisable to share the product with other users. The product should only be used by care staff for people with physical and /or mental disabilities. This product should only be used by people who can read and follow the instructions without struggling physically or mentally. The adhesive lotion has been specially developed for medical compression stockings, but it can also be used for other knitted compression garments with top bands, knitted edges or textiles. The lotion indicated cannot be guaranteed if the product is used incorrectly or the wrong amount is applied. After applying the product, hands must be washed thoroughly with water to remove product residue. We recommend that you wash the stockings regularly to remove product residue.
N.B.: do not use if the skin is inflamed or infected, if you have open wounds or weeping dermatoses or if you have an intolerance to any of the ingredients. Do not use on areas of the skin that are undergoing medical or cosmetic treatment (medical creams, ointments, plasters, wound dressings, compresses, nicotine patches, lotions, body oils, make-up products, perfumes, hair removal creams / oils, etc.). Do not apply to piercings, jewellery accessories, tattooed skin or injection tubes. Speak to your doctor before taking medication that could affect skin reactions (glucocorticoids, antiallergics, topical immunomodulators, etc.). The use of the adhesive lotion is not recommended if you have a skin condition such as neurodermatitis. In the event of an intolerance reaction, please remove the stocking and wash the skin thoroughly with water. Consult a doctor immediately in the event of an accident or feeling unwell (if possible, show the instructions for use or safety data sheet). Take off the deodorant!
Information about reporting serious incidents: in the event of a complaint with regard to the product, please contact your specialist medical retailer directly. Only serious incidents that might lead to a serious deterioration of the state of health or to death should be reported to the manufacturer and the competent authorities in the member state. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).
Hazard warnings: keep away from children, do not drink or eat the product or absorb via mucous membranes and avoid contact with the eyes.

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